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Thank you for choosing the Amplicord D1 digital electrolarynx, a mix of innovation and design to best serve your communication needs.

EN

Contents of the Box [A]

- | | |
|--------------------------------|----------------|
| ❶ Amplicord D1 | ❺ Slip Case |
| ❷ Power charger | ❻ User's Guide |
| ❸ 9 Volts rechargeable battery | |
| ❹ Security Strap | |

Product Description [B]

- | | |
|----------------|-----------------------------|
| ❶ Device Head | ❺ Power charger port |
| ❷ LED | ❻ Strap hook |
| ❸ Joystick | ❼ Battery Compartment Cover |
| ❹ Power switch | ❽ Strap hook |

Getting Started with your Device [C]

Slide the battery cover to remove it, by applying a light pressure on the three notches.

Insert the battery, matching it to the + and - symbols on the bottom of the battery compartment.

Attach the strap.

Close the battery compartment, making sure it is properly shut.

Recharging the Battery [D]

Make sure that the device is switched off, by checking that the power switch is in the top position.

Connect one end of the charger to a wall plug, and the other end to the power charger port at the bottom of your device.

The LED will turn red: the battery is now charging.

It should take 8/10 hours to fully charge the battery.

Instructions for Use [E]

Turn on the device by sliding the switch to the lower position. The device will vibrate lightly and the LED will blink green every 2 seconds. Should the battery levels be low, the LED will blink green 8 times, in rapid succession.

Apply the device onto the softer part of the neck, with a light and even pressure. The device will automatically begin functioning upon contact with the skin.

Test the device by vocalising the letter **A - E - I - O - U**.

Adjust the tone of the voice created by the device, moving the joystick to the left to decrease it or to the right to increase it.

Adjust the volume of the voice created by the device, moving the joystick downwards to decrease it or upwards to increase it.

Once you have reached the desired tone and volume, press and hold the joystick for about 1 second in order to save your preferred settings. The green light will stop blinking for a few seconds.

Device Maintenance

Maintenance will be carried out by the dealer. The device may be cleaned as often as needed, using a piece of cloth and some rubbing alcohol (surgical spirit).

Warranty

The Amplicord D1 carries a 2-Year Limited Warranty (24 months) for each of its components, valid from the Date of Purchase. The device is guaranteed against manufacturing defects (defined as a defect in materials or workmanship). The warranty states that a free replacement and/or repair will be provided for any faulty components, during the applicable warranty period. This warranty will lose validity should the device show signs of manumission, or if the defect is a result of improper use or accidental drops or falls. The power charger and the rechargeable battery are excluded from this warranty.

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