User Manual

Please read the operating instructions carefully, particularly the safety precautions, before using the appliance.

ILLUSTRATED GUIDE

1. Status Display

2. Flash Button

3. Power Socket

4. Rear Air Outlet

5. Outlet Fan

6. Power Connector

7. Power Cable 180cm

8. Power Supply 110~220v
9. Energy Ignition

9. Energy Level Button +

9. Energy Level Button -


10. Contact Sensor

11. Flash Window

12. HR Lamp

13. SR Lamp (optional)

14. AC Lamp (optional)

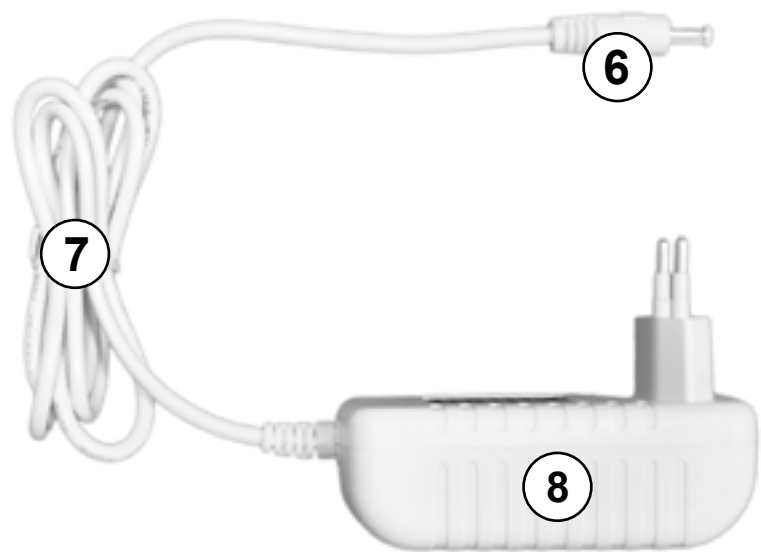D LIGHT 360 USER MANUAL

Dear Light 360 Customer thank you for purchasing this product. We are certain that you will appreciate the quality and reliability of this device, designed and manufactured by putting customer satisfaction in the foreground.

Please read the operating instructions carefully, particularly the safety precautions, before using the appliance. Keep this manual along with the relative illustrative guide, for the entire life span of the appliance, for consultation purposes. In case the appliance is transferred to a third party, also consign the entire documentation. Should certain parts of this booklet be difficult to understand or should doubts arise, contact

the company before using the product, at the address indicated on the last page.

INDEX

Safety Warnings	Pag. 2-3
Contraindications	Pag. 4-5-6
Other Safety Precautions	Pag. 6-7
Installation	Pag. 8
Operating Modes	Pag. 9
How to Prepare Treatments	Pag. 10
Epilation Treatment Procedure	Pag. 11
Photorejuvenation Treatment Procedure	Pag. 12
Acne Treatment Procedure	Pag. 13
Possible Side Effects	Pag. 14-15
Treatment Areas	Pag. 16
Patch Test	Pag. 17
Maintenance	Pag. 18
Problems and Solutions	Pag. 19
Technical and Environmental Specifications	Pag. 20

SAFETY WARNINGS

- This appliance is not intended for use by anyone under the age of 16; by persons with reduced physical, sensory, mental capabilities, or by inexperienced persons. Unless they have been given supervision or instruction concerning safe and proper use of the device by a person responsible for their safety. Children should not use, play with, maintain or clean the devise.
- If you are using the appliance in bathroom premises, disconnect the plug from the socket after use, proximity to water is also dangerous even the appliance is switched off.
- In order to ensure better protection, it is advisable to install the power circuit feeding the bathroom a residual current device, the residual current should not exceed 30 ma (Ideal 10 ma). Ask your installer for advice.

CAUTION! Do not use this appliance in the vicinity of bathtubs, showers, sinks or other containers containing water! Do not immerse the appliance in the water!

- After removing the appliance from the packaging, make sure that the product is complete and in perfect condition. Check the integrity of the supply, the design and any presence of transport damages. In case of doubt, contact the authorized service department.
- Before connecting the appliance, ensure that the power supply voltage corresponds with the rating quoted on the equipment identification nameplate. The nameplate is located on the appliance's power supply.
- Do not expose the appliance to moisture or atmospheric agents (rain, sun).
- Always disconnect appliance from the power supply when not in use and before cleaning or servicing.
- This appliance must be used only for the purpose for which it has been designed as a pulsed light fixture for home use. Any other use is considered improper and therefore dangerous.
- This device is not suitable for use in the presence of flammable anesthetic mixture with air or oxygen or nitrous oxide.
- Do not use the appliance with wet hands, damp or bare feet.

- Do not pull the power cord or the appliance to unplug it from the power socket.
- In case of appliance failure or malfunction, switch it off and do not tamper with it. For repairs, please contact the D Light technical support center.
- If the power cord is damaged, it must be replaced by an authorized technical service center, in order to prevent any risk.
- The appliance can be used only on human hair. Do not use it on animals.
- Do not obstruct the ventilation holes.
- Do not use appliance if it is cracked or broken..
- Do not use the appliance if the light is damaged, or the surface treatment is cracked, chipped or missing.
- Never look directly at the light coming from the treatment area.
- Never point the treatment surface by trying to emit a light impulse from an open space.
- In case of side effects or complications, it is advisable to contact your doctor or dermatologist.
- Protective eyewear is highly recommended when using the appliance.

CONTRAINDICATIONS

Important Safety Information Read before use

This appliance is not intended to be used by anyone. Read and take into account the information included on the following paragraph before use. For more information and personal advice you can visit the website www.dlight-ipl.com

If you are unsure if the appliance is safe for you to use, please ask your doctor or dermatologist

DO NOT USE DEVICE IF ANY OF THE FOLLOWING CONDITIONS IS PRESENT AT THE TIME OF TREATMENT:

- Pregnant or breast-feeding
- If you were exposed to strong sunlight or an artificial tanning machine during the past 28 days.
- If you have a tattoo or permanent makeup on the area to be treated.
- If you have dark brown or black spots, such as large freckles, birth marks, or warts on the area to be treated.
- If you have eczema, psoriasis, lesions, open wounds or active infections, such as cold sore in the area to be treated. Wait for the affected area to heal before using the appliance.
- If you have a history of keloidial scar formation, a known sensitivity to light (photosensitivity) or are taking medication that makes the skin more sensitive to light, including anti-inflammatory agents (such as aspirin, ibuprofen, paracetamol, acetaminophen), tetracyclines, phenothiazines, thiazide, diuretics, sulphonylureas, sulfonamides, DTIC, fluorouracil, vinblastine, griseofulvin, Alpha-Hydroxi Acids (AHAs), Beta-Hydroxi Acid (BHA), Retin-A ®, Accutane ® and/or topical retinoids.
- If you have abnormal skin conditions, such as vitiligo or albinism.
- If you have varicose veins in the treatment area.
- If you are currently or have recently been treated with Alpha-Hydroxi acids (AHA), Beta-Hydroxi acids (BHA), Retin-A ,topical retinoids or azelaic acidH.
- If you have abnormal skin conditions, caused by diabetes, for example, or other systematic or metabolic diseases

- If you have been treated with Accutane ® (isotretinoin) within the past 6 months..
- If you have been taking on steroid regimen within the past 3 months.
- If you have a history of herpes outbreaks in the area of treatment, unless you have consulted your physician and received preventive treatment before using the appliance.
- If you suffer from epilepsy.
- If you have an active implant, such as a pacemaker, an incontinence device, insulin pump, etc.
- If you have a disease related to photosensitivity, such as porphyria, polymorphic light eruption, solar urticaria, lupus, etc..
- If you have a history of skin cancer or areas of potential skin malignancies.
- If you have received radiation therapy or chemotherapy treatments within the past 3 months.
- If you have any other condition which in your physician’s opinion would make it unsafe for you to be treated. In case of doubt, consult with your physician.

SKIN TONE

The appliance removes unwanted hair by selectively addressing hair pigment. Varied quantities of pigment (melanin) are also present in the surrounding tissue of. The quantity of pigment in a particular person’s skin, which is manifested by their complexion, determines the degree of the risk they are exposed to using any method of light-based epilation.

Before using the appliance, compare your complexion with the skin tone chart shown on the product packaging

Do not use the appliance on a naturally dark complexion!

Treating the dark skin with the appliance may cause sunburn, blisters and skin color changes (hyper or Hypo-pigmentation).

TANNED SKIN

Tanned skin particularly following sun exposure, contains large amounts of melanin. This applies to all skin types and all complexion, including those who do not seem to tan quickly. The presence of large quantities of melanin exposes the skin to higher risk when using any method of light based treatment.

Avoid exposure to sun and/or tanning lights for 4 weeks before using the appliance!

EYE CAUTION

**Never use the appliance around or near the eyes!
The appliance can be used for the treatment of facial area.
Extra caution should be taken during the treatment to avoid eye area.**

NIPPLES GENITALS AND LIPS

Do not use the appliance on nipples or genital organs (male or female) and on the lips.

OTHER SAFETY PRECAUTIONS
ENERGY LEVEL

Choose your energy levels CAREFULLY! Energy level refers to the light pulse that is projected on the skin during use, from the lower level to the highest level. The notches on the status display (1) indicate the energy level at which the machine is set. As the energy level increases, it improves results as well as the risk of side effects (See “Possible Side Effects” below).

Always begin treatment using the lowest energy level!

Only if you experience little or no discomfort during and after use of the device energy at the lowest energy setting, you can increase the energy level, and so on for each subsequent session. For detailed instructions on energy level setting, see “Energy Level”.

- Always shave the area to be treated and make sure the skin is clean and dry before using the appliance.
- Cover tattoos, lips and nipples before applying the appliance.
- Cover dark brown or black spots, such as large freckles, birth marks, moles or warts before using the appliance.
- Never use flammable liquids such as alcohol (including perfumes, sanitizers, or other applications containing alcohol), or acetone to clean the skin before using the appliance.
- Do not use the appliance on any body part where you might later want hair.

PATCH TEST

Before you begin the treatment, it is recommended that you perform a patch test to identify your skin's highest level of tolerance to the pulsed light. For instructions on how to perform a patch test, see “Inspection before use” section.

DURING TREATMENT


Not treat the same area of skin more than once per session! Do not repeat the spots.

- If blisters are formed on the skin, or if the skin burns, IMMEDIATELY STOP USING THE APPLIANCE.
- Always make sure that the treatment surface points toward the skin, and is completely in contact with it.
- Remove the surface treatment (11) from the skin if it becomes excessively hot.

INSTALLATION

1) Assembling the Lights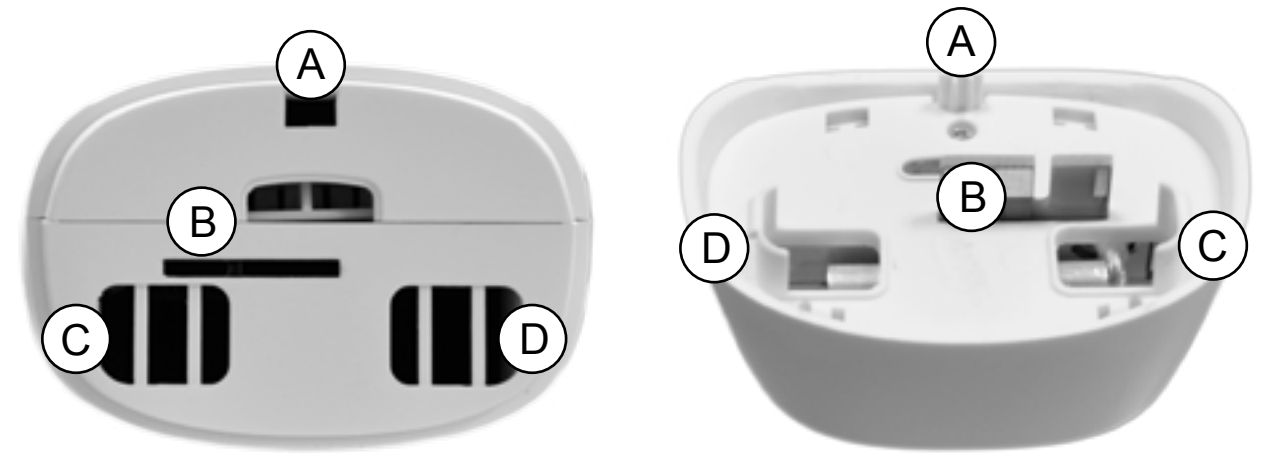

Pay attention to the direction of insertion. There are 3 plastic pivot hinges (A C D) found behind the light and an electronic card (B). During installation of the light, be careful to match these elements with their corresponding housings on the machine's attachment. Installation and removal of the light does not require equipment and is hand-made when the Machine is off and disconnected from the power socket, note that the lights are dismantled by pulling outward on one side as indicated in the drawing on the side, do not remove the light by pulling horizontally because it would risk damaging the plastic.

2) Insert the Power Connector (6) in the power socket (3)

TYPES OF TREATMENT

Depending on the type of light mounted, the appliance automatically selects the correct functional mode.

- **HR** for Epilation Mode (**H**air **R**emoval)
- **SC** for Photo Rejuvenation Mode (**S**kin **C**are)
- **RA** for Acne Mode (**R**emove **A**cne)

1) HR MODE - EPILATION

Inserting the HR Light (yellow optical filter color), D Light 360 automatically activates the Epilation mode and shows HR on the status display. In HR mode, you can choose between 5 power levels and whether to operate a single or sliding flash. The wavelength of this light is between 510nm ~ 1200nm and specifically hits hair follicles.

2) SC MODE - PHOTO REJUVENATION (SR)

Inserting the SR light (orange optical filter color), D light 360 automatically activates photorejuvenation mode and shows SC on the status display. In SR mode, you can choose between 5 power levels and whether to operate a single or sliding flash. The wavelength of this light is 560nm, specifically treats blemishes such as skin discoloration, small thin wrinkles, grayish/off color and stimulates the production of new collagen that replenish the skin, improves texture and overall tone.

3) RA MODE - ACNE (AC)

Inserting the AC light (Red optical filter color), D Light 360 is automatically activated in Acne mode and shows RA on the status display. In AC mode, you can choose between 5 power levels and whether to operate with single or sliding flash. The wavelength of this light is 430mm, specifically kills bacteria causing acne, also dried heat of the pulsed light accelerates healing.

HOW TO PREPARE THE TREATMENTS

Before starting treatments, prepare the treatment area by thoroughly cleaning the skin and remove makeup. Hair must be shaved with a razor or a hair clipper, it is preferable to use a clipper as you immediately obtain an optimal length of 1mm, and maximizes the absorption of pulsed light (in epilation treatment), using the clipper will also avoid any skin irritation.

Given that the pulsed light is absorbed by dark colors, it is necessary to protect small, dark birthmarks and tattoos, cover it with white eyepencil makeup, or with a specific product like the D Light Patch Pro.

Do not let the pulsed light pass on large or irregular areas! In the presence of doubt, ask your dermatologist before initiating treatments!

Do not use the appliance on naturally dark complexion or on tanned skin!

Do not use the pulsed light on varicose veins and any type of skin lesions

SELECT ENERGY LEVEL

It is always advisable to begin at the lowest energy level and gradually increase to find the right intensity according to your own tolerance. The energy level is selected by repeatedly pressing the ignition button (9) after the machine has been started.

As the energy level increases, so does the results, as well as the risk of possible side effects.

If blistering occurs on the skin, or if you feel burning, IMMEDIATELY STOP using the device.

EPILATION TREATMENT

The D Light 360 epilation treatment is suitable for all body areas that have hair except the scalp, male beard, eyebrows, genital and perianal areas.

Any birthmarks or tattoos on the area to be treated must be covered before starting the treatment, to protect them, you can use an opaque white makeup pencil or a specific product like the D light Patch Pro. Before starting the treatment, it is advisable to wear certified laser safety glasses or pulsed light safety eyewear, as an alternative, you can close your eyes when the flash is emitted. Define the area to be treated (can also help by drawing a grid with a white makeup pencil) and select the appropriate energy level of the skin type when you are ready.

It is always advisable to start at the lowest energy level and depending on your tolerance, if you experience no discomfort, you can increase the energy level, treatment is more effective at highest energy level, but the flash takes longer to load. You can choose between two modes of operation: single flash or scroll. Using the single flash mode, a flash is emitted everytime the Flash button (2) is pressed. The scroll mode is activated by holding the Flash button (2) down for 3 seconds, then you can release it and the device will emit flash repeatedly whenever the Flash window will come into contact with the skin. To turn off scroll mode, just press the Flash button (2) once and the appliance resumes to function on single flash mode.

When the Flash window is correctly in contact with the skin and the machine is ready for the Flash, the display starts flashing. It is recommended to radiate 1, maximum 2 times on each treated area then proceed and move to the next area.

EPILATION TREATMENT CYCLE

For the first 3 months, it is advisable to proceed with 1 treatment per week or every 2 weeks if you are also doing photorejuvenation or acne treatments on the same area, it is important to spend at least one week between other treatments by alternating Epilation to the others. The skin should be hairless after finishing the first 3 month cycle, therefore there will only be maintenance sessions needed when the hairs begin to reappear (usually a session every 2 or 3 months, depends on the of individual growth rhythm).

PHOTO REJUVENATION TREATMENT

The typical areas of photorejuvenation treatment with D Light 360 are the face, the neckline and back of the hands. With photorejuvenation treatment (SR) it is possible to attenuate the first signs of aging by stimulating the production of new collagen, which renews, firms and replenishes the skin, it is also be possible to lighten discolorations, overall skin texture and brighten the complexion in case of a grayish or washed tone.

Any birthmarks or tattoos on the area to be treated must be covered before starting the treatment, to protect them, you can use an opaque white makeup pencil or a specific product like the D light Patch Pro. Before starting the treatment, it is advisable to wear certified laser safety glasses or pulsed light safety eyewear, as an alternative, you can close your eyes when the flash is emitted. Define the area to be treated (can also help by drawing a grid with a white makeup pencil) and select the appropriate energy level of the skin type when you are ready.

You can choose between two modes of operation: single flash or scroll. Using the single flash mode, a flash is emitted everytime the Flash button (2) is pressed. The scroll mode is activated by holding the Flash button (2) down for 3 seconds, then you can release it and the device will emit flash repeatedly whenever the Flash window will come into contact with the skin. To turn off scroll mode, just press the Flash button (2) once and the appliance resumes to function on single flash mode.

When the Flash window is correctly in contact with the skin and the machine is ready for the Flash, the display starts flashing. It is recommended to radiate once, maximum 2 times on each treated area then proceed move to the next area.

PHOTOREJUVENATION TREATMENT CYCLE

It is advisable to have 1 treatment per week until the results are achieved.

If you are taking epilation or acne treatments on the same area, it is important to spend at least one week between photo rejuvenation treatment and other treatments. Once you have reached the desired results you can suspend the weekly treatments and schedule maintenance treatments according to individual needs.

ENGLISH

ACNE TREATMENT

The D Light 360 Acne treatment can be used in all areas of the body where blemishes are present, pay particular attention and wear protective glasses when treating areas near the eyes. The acne light works on a specific wavelength to kill bacteria that causes acne (Propionibacterium acnes), also the heat dry pulsed light boils accelerandone the healing and narrowing the pores by preventing the new problem to arise.

It is advisable to avoid treatment on birthmarks or tattoos, if these are present on the area to be treated, you can use an opaque white makeup pencil or a specific product like the D light Patch Pro to protect them. Before starting the treatment, it is also advisable to wear certified laser safety glasses or pulsed light safety eyewear, as an alternative, you can close your eyes when the flash is emitted. When ready, define an area to be treated (can also help by drawing a grid with a white makeup pencil) and select the appropriate energy level of the skin type.

You can choose between two modes of operation: Single flash or scroll.
To use the single flash mode, a flash is emitted every press of the Flash button (2). Holding the flash button (2) down for 3 seconds activates the scroll mode.
When you release it, the device will emit a flash then the Flash window enters into contact with the skin; To turn off scroll mode, just press the Flash button (2) once and the appliance resumes to operate in single flash mode.

When the Flash window is in correct contact with the skin and the machine is ready for the Flash, the display starts flashing. It is advisable to radiate once, maximum 2 times each, then proceed and move to the next area.

CYCLE OF ACNE TREATMENTS

It is advisable to take 1 treatment per week until the desired results are achieved, if you are also taking epilation or photorejuvenation treatments, it is important to spend at least one week between treatment and other treatments by alternating acne treatment. Once you have reached the desired results you can suspend weekly treatments and schedule maintenance treatments according to individual needs.

ENGLISH

AFTER TREATMENT

After treatment, do not expose the skin to the sun and/or tanning lights. For the following two weeks after each treatment session, in case of exposure to the sun, protect the treated skin with a minimum of SPF30 cream.

POSSIBLE SIDE EFFECTS

When used in accordance with the instructions, side effects and complications associated with the use of the appliance are rare. However, any cosmetic procedure, including those intended for household use, involve some risk. Consequently, it is important to understand and accept the risks and complications that may occur with designed pulsed light epilation systems for household use.

NEGATIVE OCCURENCE	PROBABILITY OF OCCURENCE	NEGATIVE EFFECT	PROBABILITY OF EFFECT
Multiple pulses accumulated one over the other or overlapped on the same skin area.	Minimum	Minimal Skin Discomfort	Minimum
		Skin Redness	Minimum
		Increased Skin Sensitivity	Minimum
		Wounds and Burns	Rare
		Scars	Rare
		Pigmentation Disorder	Rare
		Excessive swelling and redness	Rare
		Infection	Negligible
		Bruises	Negligible

Minimal Skin Discomfort

Despite the domestic pulsed light treatments are generally tolerated very well, most users try a annoyance during use, generally described as a slight warm sensation or a pinch in the treated skin areas. This sensation can be an alert during application or for a few minutes after application. Any sensation beyond this minimal annoyance is not normal and it means that you need to interrupt the use because your skin is not able to tolerate epilation treatment, or the energy level is too high.

Skin Redness

Skin may be reddened immediately after using appliance or during the following 24 hours. Redness generally disappears within 24 hours. Consult your physician if the redness does not disappear within 2 or 3 days..

Increased Skin Sensitivity

The skin of the treated area becomes more sensitive and can be dry or scaly.

Skin wounds and burns

Burns or wounds may occur after application. They may take a few weeks to heal, and, in very rare cases, may leave a visible permanent scar.

Scars

Although it is very rare, permanent scars may be formed. Generally when scars are formed, they have a flat lesion and white skin (hypertrophic). However, the scar may be very large and red (hypertrophic) or large and extends beyond the margins of the wound itself (keloid). Subsequent aesthetic treatments can be necessary to improve the appearance of the scar.

Pigmentation Changes

In some rare cases there is a risk of temporary pigmentation (greater pigment or Brown bleaching) or IPO-pigmentation (whitening) in the surrounding skin area leeping. The risk of skin pigmentation change is higher for people with dark skin complexion.

Redness and excessive swelling

In some rare cases the skin can become very red and swollen. This is more common in sensitive areas of the body. Redness and swelling should decrease between 2 and 7 days later, it must be cured with frequent applications of ice. You can gently wipe the area, but avoid exposing to the sun.

Infection

Infection of the skin is very rare but it is a possible risk due to scalding or injury caused by the device.

Bruises

Very seldom use of the device may cause blue-violet bruises that may last from 5 to 10 days.

NOTE:

- Treating the same body part more frequently than recommended does not improve results. Due to hormonal and physiological changes or other type, hair follicles, latents can become active. Hair removal sessions or occasional maintenance treatment can therefore be necessary.
- Individual reaction depends on hair type and biological factors that can affect the hair growth cycle. Compared to the recommended epilation program, some people may react faster or slower.
- The appliance is not effective on very clear hair, on naturally white, gray, light blond and red hair (see color scale on the packaging). If the hairs of your body are one of these colors, the device will not be effective.

INSPECTION BEFORE USE

ENERGY LEVEL

he appliance can be set to 5 energy levels, from 1 (the lowest) to 5 (the highest), represented by the 5 triangles arranged like a fan on the status display (1). The energy increases and decreases repeatedly pressing energy level button (9) Increasing the power of treatment increases the effectiveness of the CIA, but also possible side effects, it is advisable to perform a patch test to identify the appropriate energy level according to proper tolerance.

NUMEBER OF DISPOSABLE SPOTS

The meter reads the number of disposable spots available to the light inserted. Spot counting is stored in an internal chip in the light, then dismantling and reassembling the light, the count is never lost, but reparts from the point where it was left. Each light has its own count, when the light is 0, it must be replaced.

ERROR MESSAGE

When the counter displays the message “error” instead of the number of disposable spots, it means that the machine cannot detect the light inserted. The problem may occur if the light is not properly inserted, or if it is extracted while the machine is running. If the light has been inserted correctly and the message persists, it means that it is defective and must be replaced.

HOW TO PERFORM A PATCH TEST

Because the skin tone varies from area to another, you should perform a patch test to each body area to be treated. First check your skin tone comparing it with the skin color chart on the instruction manual and packaging.

- Identify a small area to be treated.
- Shave the area to be treated.
- After having set to the lowest level, apply the handpiece to the area and emit the first pulse. If you do not feel any kind of discomfort, increase the energy to the next level, move the handpiece to the adjacent part of skin and emit a new flash.

Increase the energy level on every part of the next skin until reaching the highest level (but it must remain bearable depending on skin type). During treatment it is possible (and normal) to feel a warm sensation on the skin. If there is a feeling of discomfort, it means that the highest energy level endured the previous one.

- After 24 hours check the area: the persistence of redness indicates that the skin tolerated the highest energy level on the previous one. Although it is extremely rare, if hypersensitivity persists for more than 24 hours, it is advisable to consult a physician.

CAUTION: The energy directly delivered affects the result, but because of the principle treatment, you should not exceed the maximum threshold that your skin is able to tolerate. This threshold is normally high for light skins and low for dark skins, because the presence of melanin darkens the skin, increases heat production. Do not exceed the highest threshold that your skin can tolerate without any kind of discomfort.

MAINTANANCE

CLEANING AND CONSERVATION

It is recommended that you clean the Flash window after each epilation session:

- Switch off the appliance by holding the ignition button down (9) and disconnect the Power plug (6)
- Gently wipe the flash window of the device (11) with a clean dry cloth. For proper cleaning of the flash area, it is highly recommended to use Kera Flux detergent
- Store the appliance in a cool dry place, away from heat and water sources.
- When the light is empty, the counter reaches zero.

NOTE:

If large stains appear inside the treatment surface (11) or if the latter is cracked or broken, do not use the appliance and contact the D Light Service center. After replacing the lamp, always set the energy level to lower than the one used normally. In the next session, increase the energy level again only if there is no abnormal discomfort during or after treatment.

Never immerse the appliance or any of its parts in water!

PROBLEMS AND SOLUTIONS

PROBLEM	CONTROL/SOLUTION
The appliance does not start	Connect to a better power socket
	Contact our customer support
The contact sensor does not detect the skin	Replace the lamp
	Contact our customer support
The appliance does not emit pulses	Check if the display does not blink when in contact with the skin
	Replace the lamp
	Contact our customer support
The device buttons are not pressure sensitive	Contact our customer support
The status displays the message “error”	Disconnect and reinsert the light
	Replace the lamp
	Contact our customer support

TECHNICAL SPECIFICATIONS

Type of Light	HR - SR - AC
Wavelength	HR 510nm~1200nm - SR 560nm - AC 430nm
Area spot	4.5cm²
Energy density	4 ~ 6J/cm²
Pulse number	300.000 HR / 300.000 SR / 300.000 AC
Energy level	1 ~ 5
Power supply	AC100V-240V,50Hz/60Hz
Rated input power	36W
Net weight	300g
Equipment size	17 x 4 x 8 cm
Package size	18 x 18 x 10 cm

ENVIRONMENTAL SPECIFICATIONS

- Operating Ambient Temperature: 10°C ~ 30°C
- Storage Ambient Temperature: -10°C ~ 55°C
- Degree of Operation Humidity:: up to 80% at 37°C
- Degree of Moisture Storage: up to 90% at 55°C
- Atmospheric Operation Pressure: 700-1060 hPa
- Maximum operating altitude: up to 3900 mt.
- Do not expose device directly to the sun or other radiation

If the appliance has been stored in a room with temperature too cold for days without being used, you will need to bring it back to 15 ° C or more before you can use.

WARRANTY CERTIFICATE

This warranty is without prejudice to any rights established for the consumer by the consumption Code (Legislative Decree No. 206/2005) and art.1519 bis and following the Civil Code. SMA S.R.L. With current headquarters in Via Bianche 336010 Carrè (vi) guarantees the buyer of the product (buyer) that under normal conditions of use the electronic appliances produced, assembled and distributed by SMA S.R.L. with "D light" brand (products) are free of material and workmanship defects and have been constructed in compliance with current regulations; The same products were thoroughly tested and subjected to stringent inspections by quality control. The present is the only valid guarantee; Nobody is authorized with Exclusion of the SMA s.r.l., to extend or modify the terms of the same, and even more to release other written or verbal warranties. In any case of product replacement, as well as the extension of the warranty, is excluded, as a result of the defect of conformity

WARRANTY PERIOD AND COVERAGE AREA

The products-intended as new products-are guaranteed for a period of 2 years from the date of purchase, which is proven by presenting the certificate and the fiscal receipt; while purchases by invoice and URP.IVA are guaranteed for a period of 1 year from the date of purchase pursuant to law. You do not accept purchase documents with modifications and/or additions. Failure to exhibit valid documents and proof of purchase in the above terms will result in the warranty and the charge of the entire buyer's repair costs. SMA S.R.L. does not respond to the loss of the warranty certificate and will not be liable to issue duplicates. After 2 years from the date of purchase, the product will no longer be covered by warranty and assistance may be given by the technical assistance centers (d light) authorised by the SMA S.R.L. That will charge the buyer with the current tariffs. Any form of warranty expires in any case after three years after the product has been eliminated from the official list of SMA S.R.L. Repairs made under warranty will not give any extension or renewal of the same.

WARRANTY SERVICES

All services covered by this warranty shall be subject to the actual defect of the product pursuant to this warranty. If the damage and/or defect is not covered by this warranty and/or the exclusion of the warranty, The buyer will be charged all costs related to verification, labor, replacement parts and transport of the product. The warranty may be asserted and will be effective only with the product is presented at the D Light Centre authorised by SMA S.R.L. or the direct seller. However, the product must be repaired at one of the light centers authorized by the SMA S.R.L. Under no circumstances shall the seller or buyer of the product be able to or by third parties other than authorized light centres, repairs or replacements of the product; In the case of infringement of this prohibition, the product will not be considered in warranty. It will be the buyer's responsibility to communicate to the D Light Center or to the direct seller of the product, any defects and vices of the same and the willingness to use the guarantee. The expenses and risks of transport to and from the aforementioned D Light centers will be charged to the buyer. The intervention of the authorized D light centres, the withdrawal of the product, its repairs, and its eventual substitution will occur within the time limits that are compatible with the service to be carried out, unless repairs and/or replacements are difficult. Any defective component replaced will be owned by the SMA S.R.L.

EXCLUSIONS FROM WARRANTY

All parts that are defective or damaged due to transport, negligence and carelessness in use, incorrect installation or maintenance performed by the buyer or by unauthorized personnel are not covered by any warranty, Improperly or otherwise from the one for which the product was constructed, i.e., due to circumstances which, however, can be traced to product manufacture defects. In any case, glass parts and accessories are excluded from the warranty. Do not prove that it is a manufacturing defect. The technical interventions inherent in the installation and/or testing of the equipment are also excluded from the product, in addition to parts subject to normal wear.

In any case, the warranty does not cover the products with:

- Serial number is unreadable, deleted modified or removed;
- Defects due to any use not complying with the detailed instructions stated in the user manual;
- Defects due to faults or fluctuations in electrical current and/or electrical system, air conditioning, Humidity or other environmental conditions;
- Defects due to damages caused by the use of accessories, products or related components which are not part of the product covered by the warranty;
- Damages and malfunctions resulting in use.


The warranty does not cover any accessories such as cartons, carrying cases, etc.

LIMITATION OF LIABILITY'

Compensation for direct or indirect damages of any kind to persons or things because of misuse or suspension of use of the product is not applied. In particular, the SMA s.r.l. declines any liability for any damages that may arise, directly or indirectly, to persons, pets or things, due to the failure to comply with all the requirements set forth in the product's user manual and concerning, above all, the warnings regarding installation, use and maintenance of the product.

INFORMATIVE EX ART. 13 URD.LGS. 196/03

The personal data communicated by the buyer in the coupon of the warranty certificate issued by the dealer or in the form submitted to the authorized service center of the authorised representative will be processed by SMA S.R.L. with a computer aid and manual tools with the sole purpose of making the buyer use this warranty and in order to facilitate the intervention of the D Light Center authorised by SMA S.R.L., as well as for managerial purposes related to the exercise of the economic activities of the SMA S.R.L. And for statistical purposes, marketing and commercial information relating to the activity carried out by SMA S.R.L. Such data may be communicated to other group of SMA S.R.L. companiesfor purposes related to the organization of the warranty service, assistance and maintenance. For the possible exercise of rights provided in art. 7 of Legislative decree. 196/2003, the buyer may contact SMA S.R.L.-Via Bianche 3, Carrè 36010 (vi)- the holder of the data processing. In any case, the buyer may immediately oppose the processing of his personal data for marketing and commercial information, by writing to Support@dlight-ipl.com

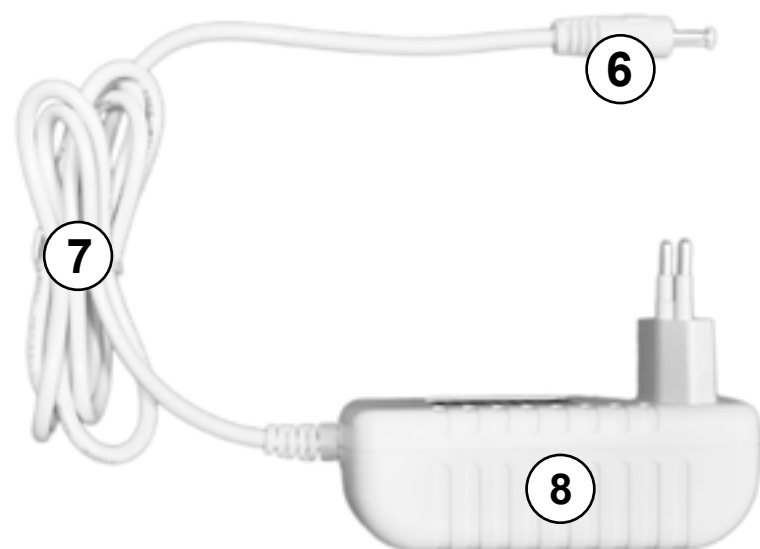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WARRANTY CERTIFICATE

This appliance has been submitted by the manufacturer to stringent testing according to international standards. We are therefore confident of its high quality. In the event that malfunctions occur, we invite you to contact the authorized service centers that will perform technical intervention with the use of original spare parts.

CLIENT DATA

Name and Surname

Street and number

CAP

City

Province

Country

Issued by:

SMA s.r.l.

Via Bianche 3
36010 Carrè (VI)
Italy

P.IVA 03547050249

English customers support
+ 39 0445 1740014

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Soporte al cliente Español
+ 34 9373 70490

Multilanguage support
support@dlight-ipl.com

CALENDAR OF TREATMENTS

[illegible]

CALENDAR OF TREATMENTS

[illegible]

CALENDAR OF TREATMENTS

[illegible]

LIGHT 360

MADE IN PRC