MIO-VEIN

Manuale d'uso
User manual

MNPG87 Rev. 3

I.A.C.E.R. Srl
Via S. Pertini 24/A – 30030 Martellago (VE) ITALY
Tel. +39 041.5401356 • Fax +39 041.5402684
e-mail: iacer@iacer.it - <http://www.itechmedicaldivision.com>

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MIO-VEIN

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User manual

MIO-VEIN

**WARNING: MIO-VEIN IS
A MEDICAL DEVICE.**

Consult your doctor before using MIO VEIN
if you suffer from health problems.

Read the user manual and the electrode position manual
before using MIO-VEIN. Read the contraindications and
warnings carefully.

CE 0476

Manufactured in compliance with the 93/42/EEC (as amended by
2007/47/EC) directive concerning medical devices by

IACER S.r.l. via S. Pertini 24/A, 30030 Martellago (Venezia) – Italy: iacer@iacer.it

USER MANUAL – TABLE OF CONTENTS

1. MIO-VEIN kit contents (photo)	Page 22
2. Purpose Contraindications and side effects	Page 22
3. Warnings and safety precautions	Page 22
4. Labelling Labels details	Page 24
5. Symbols	Page 25
6. How to use MIO-VEIN Preliminary instructions – Operating instructions	Page 26
7. Battery installation and charging	Page 29
8. MIO-VEIN maintenance and storage	Page 30
9. Resolution of problems and questions	Page 30
10. Venous pump stimulation	Page 31
11. Electrodes position and intensity	Page 32
12. Self-adhesive electrodes	Page 33
13. Technical specifications	Page 34
14. Warranty	Page 34

1. MIO-VEIN kit contents

2. Purpose.

The MIO-VEIN electronic stimulator is a medical device specifically intended for adults with adequate knowledge of the device acquired from reading the manual for domestic use. MIO-VEIN is used to apply electrical microimpulses, capable of stimulating calf muscles and helping the correct venous circulation.

Contraindications.

This device must not be used by people with a pace-maker, epileptics, people with a high temperature, anxious people or people suffering from heart disease, phlebitis, thrombophlebitis or serious illnesses, abdominal or inguinal hernias, in case of serious lesions of the area to be treated.

In the event of injury, muscle stress or any other health problem consult your doctor before using the device and only use it under medical supervision.

Side effects

No significant side effects are known of. Some particularly sensitive people could report skin redness in the area where the electrodes are positioned: the redness usually disappears a few minutes after the end of the treatment. Should the redness persist please consult a doctor.

In rare cases, stimulation carried out in the evening can cause some people to experience difficulty in falling asleep. If this occurs, suspend the treatment and consult a doctor.

3. Warnings.

MIO-VEIN has been designed and manufactured to be operated exclusively with the internal rechargeable Ni-Mh 800 mAh batteries.

- The device must not be used in the presence of patient monitoring equipment.
- Do not use MIO-VEIN with electrosurgical or shortwave or microwave therapy equipment or other equipments that send electrical impulses into the body.
- Use of the device is prohibited to persons known to be of unsound mind, or suffering from sensibility disorders, to permanently or temporarily disabled persons unless assisted by qualified personnel (e.g. a doctor or therapist).
- Use of the device is prohibited to persons younger than 15 years or not adequately educated about device use through adult persons.

- Use of the device is prohibited in the presence of signs of deterioration of the device itself, cables and electrodes.

Should any foreign materials penetrate the device contact the retailer or manufacturer immediately. If dropped down, check that the housing is not cracked or damaged in any way; if so, contact the retailer or manufacturer.

Should you notice any changes in the device's performance during treatment, interrupt the treatment immediately and consult the retailer or manufacturer (patients being treated in a centre must also be informed of the event).

- Use of the device is prohibited in combination with other medical devices.
- Use of the device is prohibited close to flammable substances/gas/explosives, in environments with high concentrations of oxygen, with aerosoltherapy devices or in wet environments (use of the device is prohibited in bathroom or shower areas).
- Consult a doctor before using MIO-VEIN with metallic osteosynthesis devices.

Muscle heaviness and cramps may be experienced in the event of prolonged muscle stimulation. Should this occur we recommend to suspend treatment with MIO-VEIN for a few days.


- It is forbidden to position the electrodes in such a way that the current crosses the heart area (e.g. a black electrode on the chest and a red electrode on the shoulder blade).
- Use of the device is prohibited with electrodes positioned on or close to injuries or cuts.
- The electrodes must not be positioned on the carotid sinuses (carotid) or genitals.
- The electrodes must not be positioned close to the eyes; make sure that the current delivered does not cross the eyeball (one electrode diametrically opposite to the other in relation to the eye); keep a distance of at least 3 cm. from the eyeball.
- When using the electrodes, follow the instructions given in this manual and on the package of the electrodes. Use single-patient electrodes, supplied exclusively by the manufacturer, and take care to avoid the exchange of electrodes between different users. MIO-VEIN has been tested and guaranteed for use with the electrodes supplied.
- Insufficiently sized electrode sections can cause skin reactions or burns.
- Do not use electrodes when damaged, even if they stick to the skin well.
- Apply only undamaged electrodes to clean and dry skin.
- Electrodes must not be used when they no longer stick to the skin.
- Pay attention to use connexion cables with children/young people: strangulation danger.
- Keep right distance between electrodes: the contact between electrodes could cause wrong stimulations or irritations/burns.

If you have any doubts regarding the device use consult your doctor.

- The device can be used by private user in a domestic environment.
- Pay attention to use of device with patients with suspected cardiac problems.
- Stimulation intensity and electrodes position should be suggested by prescriber doctor.
- Treatment efficacy depends on the patients selection by qualified persons. +
- If the stimulation is uncomfortable decrease intensity. If the problem persists consult a doctor.
- Do not use the device while driving a vehicle or controlling machineries/equipments.
- Do not mix connexion cables up with earphones or other devices and do not connect the cable to other equipments.
- Do not use pointed or sharp objects on the device keyboard.
- Use only accessories supplied by device manufacturer.


Warning: device supplies current over 10 mA.

4. Labelling.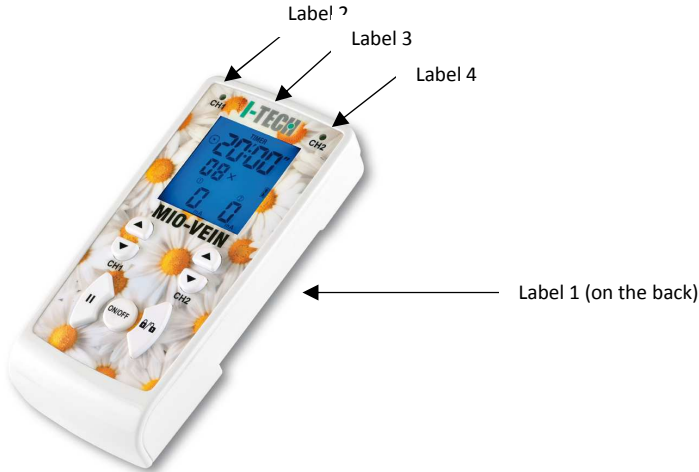

Labels details.

Label 1

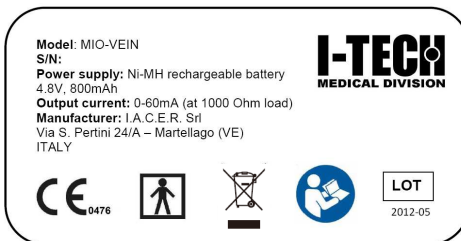
Label 2

Label 3

Label 4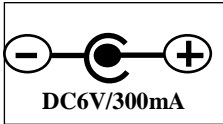


5. Symbols

	Refers to operating instructions
	Compliance with Directive 93/42/EEC (MDD) and following modifications
	Applied part type BF
	Attention
	Product subject to WEEE regulations concerning separate waste collection of electronic equipment
	Manufacturing date (month/year)

Expected lifetime: 5 years

6. How to use MIO-VEIN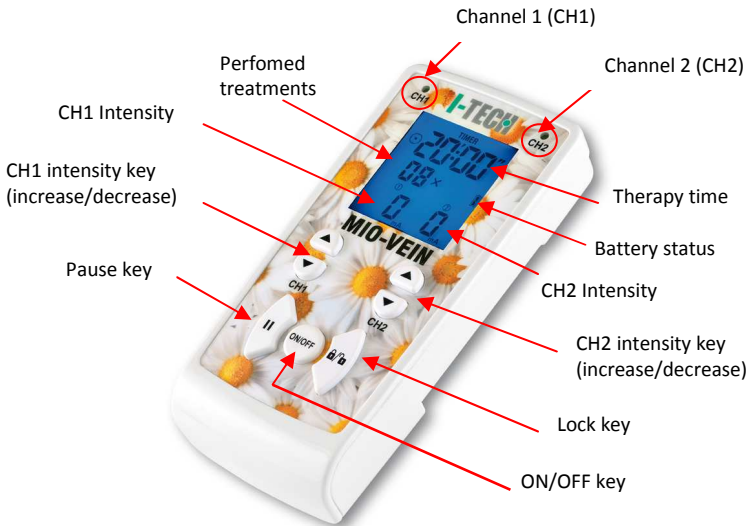

Preliminary instructions.

Before start using MIO-VEIN clean the skin in the treatment zone; connect the cables to the electrodes; place the electrodes on the skin (read chapter 11 Electrodes Position and intensity); connect the cables to the outputs (CH1 and/or CH2) and switch on MIO-VEIN.

Warning: make sure that MIO-VEIN is switched off before disconnecting the electrodes at the end of the treatment.

Operating instructions (N.B. read the whole manual before using the equipment)


1. Switch on MIO-VEIN pressing on **ON/OFF** button.
2. The display shows therapy time (20 minutes), on the middle left side the number of treatments have been executed and on the right the battery status icon.

3. Check battery status: follow the instructions of chapter "Battery charging".
4. Start the program pressing ▲ button of channel 1 or 2 and then increase the intensity; icon [🌀] starts blinking on the display.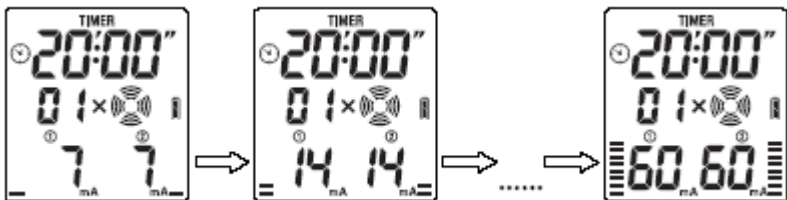

5. The display will show the remaining therapy time, while it will be possible to increase the intensity pressing the ▲ button of the selected channel till the personal tolerance limit (stimulation comfort). If the intensity is too high or uncomfortable, decrease it with ▼ button.
6. When intensity reaches a value of 7mA, a bar icon (growing intensity) will be showed on the display for each channel. The maximum intensity is 60mA and it is increased by 1mA each step;

7. The program is divided into 2 phases: a sound signal informs the phase changing. The number of the phase in progress (respectively **01** and **02**) will be showed on the middle side of the display.
8. A sound signal informs of the end of the program, MIO-VEIN will return to the main menu, decreasing the intensity to zero.
9. The therapy can be suspended pressing the key : the display will show a flashing icon on the upper right side  To restart therapy pressing again 
10. To switch OFF MIO-VEIN keep pressed **ON/OFF** key.

WARNING: MIO-VEIN will enter into “sleep mode” if not press any key for 2 minutes to preserve batteries.

Key lock

The device is provided with key lock function to avoid accidental modifications to parameters during the treatment.

- Key lock is automatically setted after around 20 seconds from the beginning of the therapy: the display will the flashing icon  on the left side and the buttons will be locked.
- To unlock the key lock function press the button : all the buttons will be unlocked and the icon on the display will disappear.

Warning: MIO-VEIN has a safety output control to check the correct connection of cables and electrodes, and also the correct transmission of the current to the patient. In case of uncorrect **connection is not possible to exceed 9mA intensity: MIO-VEIN will stop erogation and return to main screening.**

Check cables and electrodes status (eventually replace them) and the correct connection to the patient. If the problem persists contact the manufacturer.

The device is provided with a function memorizing the treatments that are performed. The treatments number is shown on the display before starting the session when device is in stand by. If the treatment is interrupted before the end, it will not be memorized.

The maximum treatments number that can be memorized is 99: over this value the memory is resetted and the counting restarts from 1.

7. Battery installation and charging.

Battery installation.

Open the battery compartment on the back side of the device, connect the battery wire to the plug inside the battery compartment.

Insert the battery and close the battery compartment.

Attention: remove the battery in case of prolonged inactivity (over two months).

Batteries have to be handled by adult persons: keep them out of children's reach.

MIO-VEIN is supplied by internal rechargeable Ni-Mh 800 mAh battery with new long-lasting technology .

The display shows when the battery is low of energy:

To proceed with the charging follow the steps below:

- Make sure that the device is switched off before charging
- Make sure that the device is not being used by patient (disconnect cables and electrodes)
- Connect the battery charger to the plug on the upper side of MIO-VEIN and connect the battery charger into the power socket. The display will show the flashing icon of battery status on the right side.

The battery must be recharged for 6/8 hours.

At the end of battery charging, disconnect the charger from power socket and from the device.

Do not immerse the battery in water or other liquids and do not expose it to heat sources.

Do not dispose of dead or defective batteries with domestic waste; dispose of in an authorized waste collection bin.

WARNING: do not leave the battery completely discharged for long periods. We suggest to recharge MIO-VEIN once a month for 6/8 hours.

We suggest to fully discharge batteries before the new charging to guarantee a longer batteries duration.

WARNING: after the first recharge, the battery could present a limited capacity. This is a normal characteristics of Ni-Mh battery. We suggest to recharge the battery after each treatment for 3 or 4 times.

Do not use the battery charger if:

- The plug is damaged or parts of it are broken.
- It has been exposed to rain or any other type of liquid.
- The components have been damaged by a fall.

Use a dry cloth to clean the battery charger. Do not open the battery charger: it does not contain repairable parts.

8. MIO-VEIN maintenance and storage

We recommend that MIO-VEIN is switched off and the cables are removed from the connectors at the end of every session. MIO-VEIN should be kept in the carry bag supplied, together with the rest of the equipment supplied with it, and carefully stored on a secure surface and protected from situations listed in the *Warnings* paragraph.

MIO-VEIN should be stored in the following conditions:

Outside of the packaging:

Temperature: from +5°C to +40°C
Maximum relative humidity: from 30% to 85%
Atmospheric pressure: from 700 hPa to 1060 hPa

Inside of the packaging:

Temperature: from -10°C to +50°C
Maximum relative humidity: from 10% to 90%
Atmospheric pressure: from 700 hPa to 1060 hPa

Cleaning: only clean the device using a dry cloth.

We recommend a functional test carried out by the manufacturer every 24 months. The manufacturer retains that the MIO-VEIN device cannot be repaired by personnel external to its own company. Any work of this nature carried out by personnel not authorised by the manufacturer will be classed as tampering with the device, freeing the manufacturer from responsibility for the warranty and hazards that the operator or user may be exposed to.

N.B. disconnect the cables before placing the electronic stimulator in the bag. If not, the cables will bend excessively close to the connectors. This may damage the cables.

9. Resolution of problems and questions.

Any type of work on MIO-VEIN must be carried out exclusively by the manufacturer or by an authorised dealer. In any event, any presumed malfunction of MIO-VEIN must be verified before sending the device to the manufacturer.

Here below are some typical situations:

MIO-VEIN cannot be turned on:

- Recharge the batteries and try to switch it on again; if it still does not switch on, check that the power supply socket is working.
- Check that the ON/OFF button has been pressed correctly (held down for at least one second).

MIO-VEIN does not transmit electric impulses:

- Check that the cable jacks have been inserted in the electrodes and that the plastic protection has been removed from the electrode.

- Check that the cables have been connected correctly (connector well inserted in the device).
- Check the electrodes status: damage or poor quality electrodes cannot properly transmit electrical impulses. Substitute the electrodes if necessary.

MIO-VEIN switches off during operation:

- Recharge the batteries and start the treatment again. If the problem persists contact the manufacturer.

Warning: MIO-VEIN has a safety output control to check the correct connection of cables and electrodes, and also the correct transmission of the current to the patient. In case of uncorrect **connection is not possible to exceed 9mA intensity: MIO-VEIN will stop erogation and return to main screening.**

Check cables and electrodes status (eventually substitute them) and the correct connction to the patient. If the problem persists contact the manufacturer.

Disposal

To protect the environment, the device, its accessories and above all the batteries must be disposed of in areas equipped for that purpose or as special waste.

Product subject to WEEE regulations concerning separate waste collection: when disposing this product, please use the designated areas for disposing electronic waste or contact the manufacturer.


10. Venous pump stimulation.

Electrostimulation consists of the transmission of electric microimpulses to the human body. Fields of application of electrostimulation are: pain therapy, muscles stimulation and beauty treatment.

Specific electric impulses are used for every one of these applications.

MIO-VEIN is an electrotherapy device for deep stimulation of calf muscles. Its stimulation reactivates calf muscular pump that is mainly responsible of venous return.

Calf muscles contraction promotes the compression of superficial and deep veins. As a consequence blood reaches easily the heart.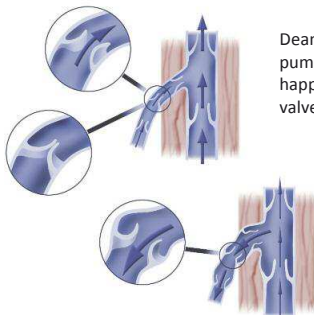

Deambulation helps venous return thanks to muscle pump that moves bloodstream back to the heart. This happens in normal conditions when veins have healthy valves and walls.

As veins walls and valves become weak blood flow slows down and stagnate (venous stasis). This causes the inversion of venous circulation.

MIO-VEIN is suggested for all people suffering from venous diseases and also in all situations that represent potentially risk factors for venous problems.

The main clinical applications are:

- Venous stasis
- Lower limbs oedemas
- Varicose veins
- Post-phlebitic syndrome
- Post-thrombotic syndrome
- Pregnancy related venous disorders
- Venous ulcers
- Leg itch after compressive stocking use

MIO-VEIN helps the correct venous circulation and consequently it promotes healthy and beauty legs.

11. Electrodes position and intensity.

WARNING!


It must be remembered that an electronic stimulator is a very effective analgesic instrument and that pain can indicate various types of medical conditions!
You are advised to read the manual carefully before using MIO-VEIN

The stimulation intensity is displayed on the MIO-VEIN screen for each channel in a scale progressing from 0 to 60 and with a bar indicator.

The bars numbers increase as adjusted intensity increases.

The oval electrodes should be positioned on each calf (sitting or lying down). The intensity should be adjusted to obtain a soft muscles contraction.

See the following photos for the correct electrodes position.

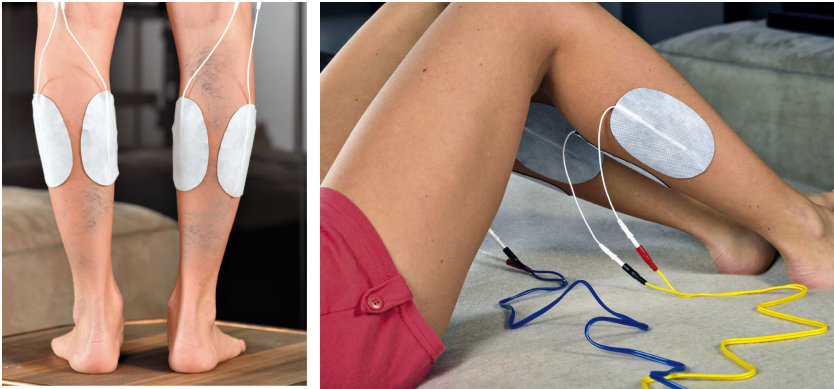12. Self-adhesive electrodes.

The position of the electrodes is shown in the previous images.

The electrodes supplied are top quality, pre-gelled and ready for use, they are particularly suitable for electrostimulation treatments.

They are very flexible making it easy to treat all areas. Remove the protective plastic from the electrode before use, position it on the skin as shown in the manual and replace the plastic after use.

Electrode duration is determined by the skin's pH value; we recommend no more than 20/30 applications using the same electrode.

Repeated use of the same electrodes can compromise stimulation safety, which is why electrodes must not be used when they no longer stick to the skin; in fact it can cause skin redness that can last for many hours after stimulation. Consult a doctor if this occurs.

For a safe use please refer to indications and warnings on electrodes package and in "Warning" paragraph.

13. Technical specifications.

<i>Risk class:</i>	Class II in accordance with the 93/42/EEC directive (amended by 2007/47/EC).
<i>Isolation class:</i>	Internally powered device with type BF applied parts, in accordance with the EN 60601-1 standard.
<i>Classification:</i>	IPX0, based on the penetration of liquids and powder; equipment not protected.
<i>Housing:</i>	ABS.
<i>Power Supply:</i>	Ni/Mh rechargeable battery 4,8V 800mAh.
<i>Limitations:</i>	equipment not suitable for use in environments with a high concentration of oxygen and/or flammable liquids and/or flammable gas; do not use with equipment for electrosurgery or short-wave or microwave therapy.
<i>Operation:</i>	equipment suitable for continuous operation.
<i>Output channel:</i>	2 channels, independent and galvanically isolated.
<i>Output current:</i>	maximum power 60mA with 1Kohm load.
<i>Type of impulse:</i>	compensated, biphasic, rectangular.
<i>Frequency:</i>	operates with 250 Hz burst impulses.
<i>Impulse amplitude:</i>	from 30 μ s to 240 μ s.
<i>Display:</i>	reflective LCD display.
<i>Control keypad:</i>	7-key keypad in ABS.
<i>Dimensions:</i>	114X68X26 mm.
<i>Weight:</i>	210 gr. including batteries.
<i>Maintenance and storage:</i>	refer to the instructions in Chapter 7 "Maintenance and storage of MIO-VEIN".

Warning: the equipment delivers current in excess of 10mA.

 0476 Produced in compliance with the 93/42/EEC (amended by 2007/47/EC) directive concerning medical devices.

14. Warranty

Make reference to the national laws for any warranty conditions by contacting the national distributor (or directly the manufacturer IACER).

I.A.C.E.R.S.r.l. SU

Sede operativa:

30030 Martellago (VE) - Via. S. Pertini 24/A
Tel +39 041 5401356 - Fax +39 041 5402684

Sede legale:

S. Marco 2757 - 30124 Venezia
Cod. Fisc./P.IVA IT 00185480274
R.E.A. VEN. 120250 - M. VE001767
Cap.Soc. € 110.000,00 i.v.
www.iacer.ve.it - iacer@iacer.it