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MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com
MADE IN P.R.C.

INDEX

1. INTRODUCTION	pag.10
2. INTENDED USE	pag.10
3. EC DECLARATION OF CONFORMITY	pag.10
4. GENERAL WARNING	pag.10
5. GENERAL DESCRIPTION	pag.11
5.1 Multifunction chairs for bathroom - codes and description	pag.11
6. BEFORE USE	pag.11
7. BEFORE ASSEMBLY	pag.11
8. ASSEMBLY INSTRUCTION	pag.11
9. SEAT	pag.11
9.1 To lift the armrest (only for RP763)	pag.12
9.2 To lower arms (only for RP763)	pag.12
10. USING WARNINGS	pag.12
11. HOW TO USE	pag.12
12. MAINTENANCE	pag.12
13. CLEANING AND DISINFECTION	pag.13
13.1 Cleaning	pag.13
13.2 Disinfection	pag.13
14. CONDITIONS OF DISPOSAL	pag.13
14.1 General conditions of disposal.....	pag.13
15. TECHNICAL FEATURES	pag.13
15.1 Spare parts	pag.13
16. WARRANTY	pag.14
16.1 Repairs	pag.15
16.2 Spare parts	pag.15
16.3 Exempt clauses	pag.15

I Class Medical Device according to Directive 93/42/EEC
and further modifications

1. Introduction

Thank you for purchasing a **MOPEDIA** multifunction chairs for bathroom Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance.

2. Intended use

The multifunction chairs for bathroom **MOPEDIA** by Moretti are intended to be used as an aid in shower and bathroom for people with walking difficulties. They include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice.

3. EC declaration of conformity

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the cane aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications.

To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I
6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with Directive 93/42/EEC to the proper authorities and further modifications

4. General warnings

- **READ** carefully the entire manual before using the device
- For a correct use of the device always consult your physician or therapist
- Make sure that the product is delivered complete and keep it packed away from heat sources
- **SERVICE LIFE**- the device use limit is defined by the wear of the parts
- **DO NOT** allow children to play or operate the device

5. General description

5.1 Multifunction chairs for bathroom - codes and description

- RP760** Fixed model
- RP761** Foldable model
- RP762** Demountable model with castors
- RP763** Steel frame with foldable armrest

The multifunction chairs for bathroom **MOPEDIA** are made of anodized aluminum (RP763 is steel), with a comfortable chair in PVC and non-slip rubber tips (except model RP762 equipped with castors). Adjustable height every 2,5 cm. They include four functions in one: a stabilizer for water rise, support for water, commode chair and shower seat. Available fixed model, foldable and demountable with castors.

6. Before use

- Always check if there is any part worn out in order to ensure the device is used in complete safety for persons and property
- Check the correct assembly of the device

7. Before assembly

Unpack the product and check that no damage has occurred during transport. In the event that the product is damaged, please contact your nearest dealer.

8. Assembly instruction

RP760 Insert the toilet support to the product which is supplied already assembled, making sure to fit correctly the support with the horizontal rear bar.

RP761 Open the sides of the frame, lift the part with the toilet support already assembled and fix the two ends to lateral parts. Next, install the rear bar.

RP762 Insert the two horizontal bars in the supports of the side parts making them block thanks to the safety clips. Insert the toilet support paying attention to fit it properly with the horizontal rear bar.

RP763 Insert the toilet support paying attention to fit it properly with the horizontal rear bar. Insert the backrest bar upwards into the holes of the rear rails making it block thanks to the safety clips.

9. Seat

Press the safety clip to adjust the height of the chairs by removing the inner part of the leg and bringing the holes to the desired height. Check that all four legs are adjusted to the same height.

9.1 To lift the armrest (only for RP763)

Swing the armrest upward inserting them in the safety clips. (fig.1)

9.2 To lower arms (only for RP763)

Press the safety clips inward and swing the armrest downward. (fig.2)

(fig.1)

(fig.2)

NOTE Before you sit down:

- Place the product on a flat and solid surface and if present, lock the brakes of the castors
- Make sure that the lock latch is secure and that the product is in a stable position

10. Using warnings

- Use caution on wet and slippery surfaces
- DO NOT use with patient who exceeds the weight indicated in the product label and in the technical features chapter of the present manual
- The chair can be used only on a hard, flat surface. Do not use the chair on stairs, inclines or declines
- Do not use the chair in a way other than that described in this manual
We disclaim any responsibility for improper use of the chair
- Avoid to leave the device in contact with water for long periods of time
- Periodically check the stability and hardness of the product as well as the status of the tips and castors. Replace worn once
- In case of doubts about use, adjustment or maintenance, contact your dealer
- Keep it away from children to avoid any possible risks

11. How to use

- Setting out to sit with your weight on the seat assembly
Do not sit by pushing on one side only
- Leaning always on both arms to get up and keep your balance
- In case of use of the chair by people with movement difficulties, help them always in the sitting or in getting up from the chair

12. Maintenance

The MOPEDIA devices by MORETTI are checked carefully and supplied with EC mark, once launched on the market. For patient and physician's safety we recommend to let check the device by the manufacturer or by laboratory authorized every 2 years.

For periodic inspection means an examination of the following parts:

- The frame of the chair and connecting parts
- Check mounting and operation of the wheels with its brake
- Check all the welding points
- Check the wear of the tips where present

In case of repair, use only original spare parts and accessories.

13. Cleaning and disinfection

13.1 Cleaning

Use only a damp cloth to remove dust, use a damp cloth with a mild soap to remove more difficult dirt. Dry thoroughly before use. Use water at a temperature not exceeding 30 ° C. Do not use washing machines jet of water and / or steam

13.2 Disinfection

If you need to disinfect the product use a common disinfectant cleaner.

NOTE Never use acids, alkalis or solvents such as acetone

14. Conditions of disposal

14.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose the bath chair in the appropriate disposal areas for recycling.

15. Technical features

Codes	Max user's weight	Adjustable height	Adjustable height seat	Base	Material used
RP760	110 kg	From 40 to 55 cm	From 40 to 55 cm		Aluminum frame
RP761	110 kg	From 64 to 74 cm	From 44 to 54 cm		Aluminum frame
RP762	110 kg	From 66 to 77 cm	From 48 to 59 cm		Aluminum frame
RP763	110 kg	From 63 to 78 cm	From 43 to 58 cm		Steel frame

15.1 Spare parts

RV7058
Spare seat and cover for commode chair **RP760**
Hole dimensions: 20,5 x 24 cm

RV7058-1
Spare seat and cover for commode chair **RP761**
Hole dimensions: 20,5 x 24 cm

**RV7058-2**

Spare
seat and
cover for
commode
chair
RP762

**RV7058-3**

Spare
seat and
cover for
commode
chair
RP763

**RV7068**

Bucket, hook
and cover
for commode
chair **RP762**

**RV7069**

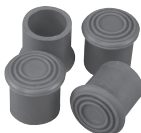
Bucket, hook
and cover
for commode
chair **RP760-
RP761**

**RV7069-1**

Bucket, hook
and cover
for commode
chair **RP763**

**RV7024**

Ø 29mm
Tips for
commode chair
RP761-RP763
4 pcs

**RV7025**

Ø 23mm
Tips for com-
mode chair
RP760
4 pcs

**RV7051**

Swivel
casters for
RP762
pairs

16. Warranty

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use.

Moretti Moretti does not guarantee the products against damage or defects in the following conditions: natural

disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

16.1 Repairs

Warranty repair: In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location . The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty

Repair not covered by warranty: A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

Non-defective device: The customer will be informed if, after having examined and tested returned product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

16.2 Spare parts

The original Moretti spare parts are guaranteed for 6 (six) months from the receiving date.

16.3 Exempt clauses

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

MORETTI

WARRANTY CERTIFICATE

MOPEDIA

Product _____

Purchased on (date) _____

Retailer _____

Address _____ **Town/city** _____

Sold to _____

Address _____ **Town/city** _____

MORETTI S.P.A.

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com email: info@morettispa.com

MADE IN P.R.C.

Via Bruxelles, 3 - Meleto
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com