MOPEDIA®

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CE I Class Medical Device

According to Directive 93/42/EEC and further modifications

1. CODES

RS975-36 Suction grab bar - 36 cm**RS975-50** Suction grab bar - 50 cm**RS975-63** Suction grab bar long/angled - 63 cm**RS975-90** Suction grab bar long/angled - 90 cm

2. INTRODUCTION

Thank you for purchasing a MOPEDIA suction grab by Moretti S.p.A. This user's manual provides some suggestions as how to correctly use the product you have chosen and gives some valuable advice for your safety. Please read through the manual carefully before using the product. Should you have any queries, please contact your retailer for advice and assistance. In case of doubts, we recommend to contact the dealer who can help and advice you properly.

Note: Check if any part has shipping damages and test before using. In case of damage, do not use the device. Contact your dealer for further instructions.

3. INTENDED USE

The grab bar by Moretti is intended to help people with movement difficulties.

CAUTION!

- Do not use the product for a purpose not indicated in this manual
- Moretti S.p.A declines all responsibilities for any consequences resulting from an incorrect use of this product and from unauthorized alteration to the frame of the product
- The manufacturer reserves the right to change the information contained in this document without previous notice

4. EC DECLARATION OF CONFORMITY

Moretti S.p.A. hereby declares, under its own exclusive responsibility, that the products manufactured and sold by Moretti S.p.A. and belonging to the bath aids category are in compliance with the applicable provisions outlined in the Medical Devices Directive 93/42/EEC and further modifications. To this end Moretti S.p.A. guarantees and declares as follows, under its own responsibility:

1. The devices in question satisfy the essential requirements of Annex I of directive 93/42/ EEC as prescribed by Annex VII of the above directive
2. The devices in question **ARE NOT MEASUREMENT DEVICES**
3. The devices in question **ARE NOT INTENDED FOR CLINICAL INVESTIGATION**
4. The devices in question are sold in **NON STERILE** packing
5. The devices in question should be considered as belonging to class I

6. **DO NOT** use/install the devices in question for any other purpose than that expressly indicated by Moretti S.p.A.
7. Moretti S.p.A. maintains and submits the documentation certifying conformity with

5. GENERAL WARNINGS

- DO NOT use this product or any available optional equipment without first completely reading and understanding this instruction manual.
- Keep the packed product away from heat sources
- The suction cup grab bar is not designed to hold or support your full body weight.
- This grab bar is designed to assist your natural motion, as when stepping into or out of a shower.
- Do not attach to moving surfaces.
- Maximum User Weight : 55 kg (120 lbs) short grab bar - 70 kg (150 lbs) long grab bar - (Lateral & Vertical); 40 kg (90 lbs) - Angled
- SERVICE LIFE- the product use limit is defined by the wear of the parts
- DO NOT allow children to play on or operate the bath chair

6. GENERAL DESCRIPTION

The suction grab bar is designed for use only on smooth and non-porous surfaces; tile, glass, porcelain or fiberglass. Not for use on wall board, wood, painted or papered surfaces. Usable in all environments where it is not permitted to drill the walls or outside the home.

7. BEFORE USE

- Always check the attachment and condition of suction cup thanks to the indicator on the handle
- Check if the frame is damaged in order to guarantee a safe use for people and things
- Check the correct assembly of the device

8. WARNINGS FOR USE

- You can not sit on the grab bar;
- If the grab bar is used in unfavorable conditions (eg. Suckers damaged or worn), the holding may disappear suddenly;
- Under water the grab bar holding decreases;
- The holding of the suction cups decreases in the presence of high temperatures and high altitudes;
- Keep the suckers away from direct sunlight, lose elasticity. It should not attach the handles to the outside surface illuminated type windows;
- If the handle remains attached in the same place for a prolonged period of time, to the removal, the underlying surface may remain blotchy.



9. ASSEMBLING

9.1 Assembling for short grab bar


The short grab bar is supplied already assembled.

9.2 Assembling for long grab bar


The long handle may be assembled in linear or angled depending on the needs of the patient.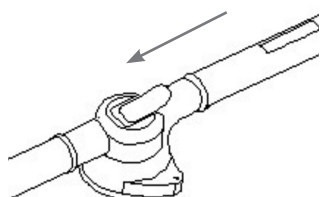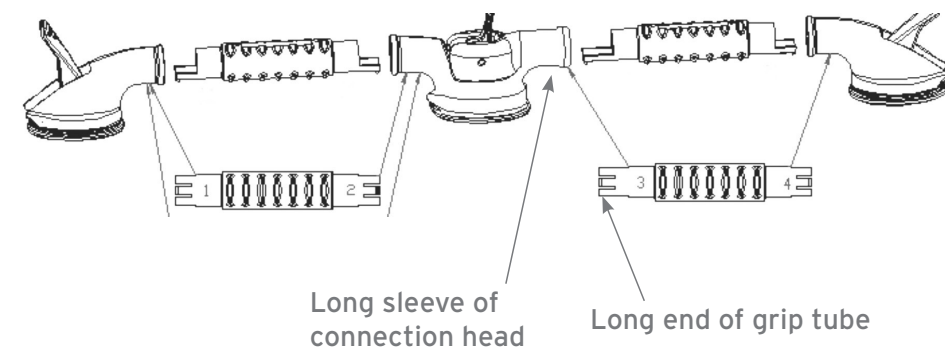


Insert the long end of grip tube to the long sleeve of connection head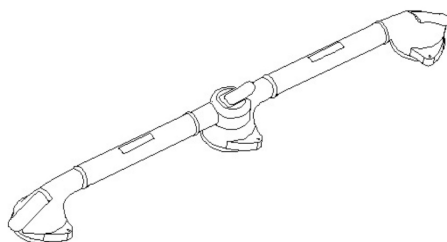

Insert the short end of grip tube to the other short sleeves of connection head.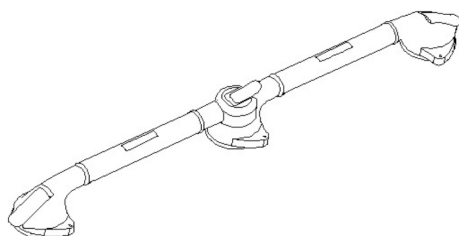

Straight

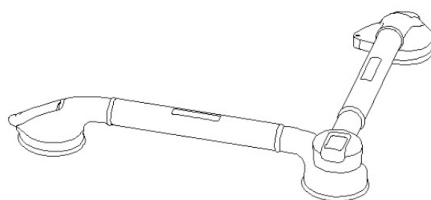Angled

9.3 Grab bar positioning

1. Thoroughly clean the surface with alcohol wipe and allow to dry and check the condition of the suction cups. Do not use if damaged, cracked or split.
2. Raise both levers; place the grab bar so that both suction cups lie firmly and completely on a smooth surface.
3. Hold one end and press on one lever with your palm, You should feel resistance as the suction cup tightens.
4. Check that the handle adhered well to the surface, otherwise reposition.

9.4 Removal

1. Lift each of the two levers.
2. If suction remains, gently lift one edge with a finger.
3. Do not use a sharp instrument which may damage the suction cup.
4. Always keep the levers flip up until next use.

9.5 Patented suction indicator

The indicator seal (patented system), consists in an alerter of the actual adhesion of the suction cup to the support thus ensuring maximum grip and safety during use.

“GREEN” means
the correct
attaching

Otherwise, it will turn into the **“RED”**
request to check the mounting
surface and re-attach again

N.B. Do not place over joints or grout lines.

10. TECHNICAL FEATURES

Each grab bar is made of ABS with suction cups made of PVC, they are mobile grab bars and compared to the fixed traditional ones, have several advantages:

- Mounting without drilling holes;
- Horizontal, vertical and angled positioning
- They can be moved at any time and can follow the evolution of the disease.

- Quick and easy assembly and disassembly
- Available in different sizes:

RS975-36 suction grab bar - 36 cm

RS975-50 suction grab bar - 50 cm

RS975-63 suction grab bar long/angled - 63 cm

RS975-90 suction grab bar long/angled - 90 cm


10.1 Material


Handle ABS


Suction PVC


Spring and rivet steel

10.2 Dimension

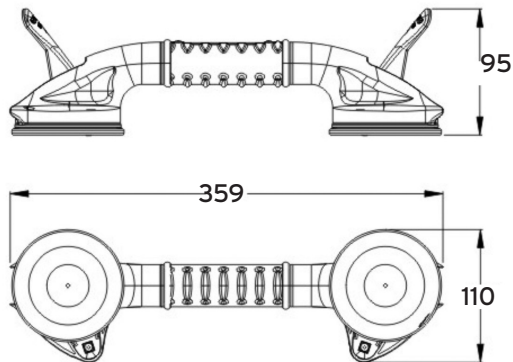Dimension RS975-36	
 Maximum User Weight	55 Kg (120 lbs) (Lateral & Vertical) - 40 Kg (90 lbs) (Angled)

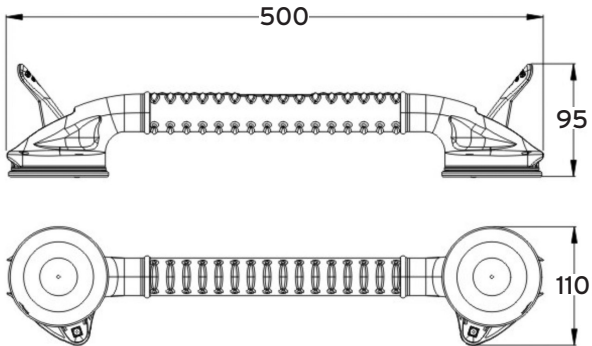Dimension RS975-50	
 Max loading weight	55 Kg (120 lbs) (Lateral & Vertical) - 40 Kg (90 lbs) (Angled)

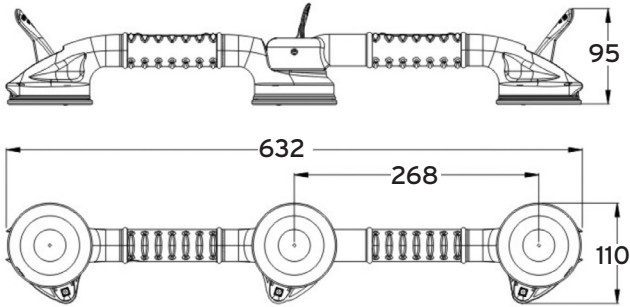Dimensioni RS975-63	
 Max loading weight	70 Kg (150 lbs) (Lateral & Vertical) - 40 Kg (90 lbs) (Angled)

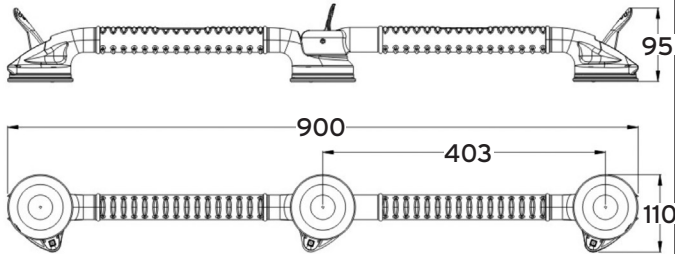Dimensioni RS975-90	
 Max loading weight	70 Kg (150 lbs) (Lateral & Vertical) - 40 Kg (90 lbs) (Angled)

11. CLEANING AND DISINFECTION

11.1 Cleaning

Use a damp cloth and mild soap. Then dry thoroughly before use.

11.2 Disinfection

If you need to disinfect the device use a common disinfectant cleaner.

NOTE Never use acids, alkalis or solvents such as acetone.

12. CONDITIONS OF DISPOSAL

12.1 General conditions of disposal

In case of disposal don't use the inserting container for municipal waste. We recommend to dispose.

The bath chair in the appropriate disposal areas for recycling.

13. SPARE PARTS AND ACCESSORIES

For spare parts only refer to MORETTI master catalogue.

14. WARRANTY

Moretti products are guaranteed from material or manufacturing faults for a period of (two) 2 years from the purchasing date, subject to any limitations and exclusions specified below. The warranty does not apply in case of damages caused by improper use, abuse or modification of the product, and the warranty will not be valid if the instructions for use are not strictly followed. The correct intended use of the device is specified in this manual.

Moretti is not responsible for damages resulting from personal injury or otherwise caused by or in relation to wrong installation or improper use. Moretti does not guarantee the products against damage or defects in the following conditions: natural disaster, not authorized maintenance or repair, damage due to power supply problems (where provided) use of spare parts not supplied by Moretti, improper use, not authorized alteration, shipping damage (other than original shipment by Moretti), or in case of insufficient maintenance as indicated in the manual.

The warranty doesn't cover components subject to wear and tear during the correct use of the device.

14.1 Repair

In the event that a product Moretti to be defective in material or manufacturing faults during the warranty period, Moretti will confirm with customer if the fault can be covered from warranty. Moretti, in its sole discretion, can repair or replace the item under warranty at a Moretti dealer or at Moretti location. The labor cost for the repair may be charged to Moretti if the repair is covered by warranty. Repair or replacement does not renew or extend the warranty.

A product not covered by warranty can be returned for repairing only after receiving prior authorization from Moretti customer service. The labor and shipping costs for a repair not covered by warranty will be fully borne by the customer or dealer. Repairs of products not covered by warranty are warranted for six (6) months from the date you receive the product repaired.

The customer will be informed if, after having examined and tested returned

product, Moretti claims that the product is not defective. In this case the good will be returned to the customer, all shipping costs will be paid by customer.

14.2 Spare parts

For spare parts only refer to MORETTI master catalogue.

14.3 Clausole esonerative

Except as expressly specified in this warranty and within the limits of the law, Moretti does not offer any representation, warranty or condition, express or implied, including any representations, warranties or conditions of merchantability, fitness for a particular purpose, non-infringement and interference. Moretti does not warrant that the use of the product Moretti will be uninterrupted or error free. The duration of any implied warranties that may be imposed by law are limited to the warranty period, within the limits of the law. Some states do not allow limitations on how long an implied warranty lasts or the exclusion or limitation of incidental or consequential damages in connection with products for consumers. In such states or countries, certain disclaimers and limitations of this warranty may not apply to you. This warranty is subject to change without notice.

**WARRANTY CERTIFICATE****Product** _____**Purchased on (date)** _____**Retailer** _____**Address** _____ **Town/city** _____**Sold to** _____**Address** _____ **Town/city** _____

Via Bruxelles, 3 - Meleto 52022 Cavriglia (Arezzo) Tel. +39 055 96 21 11

www.morettispa.com **email: info@morettispa.com****MADE IN P.R.C.**

This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

This image shows a single sheet of white paper with horizontal blue or grey ruling lines. The lines are evenly spaced and run across the width of the page. There are no margins, text, or other markings on the paper.

Via Bruxelles, 3 - Melegnano
52022 Cavriglia (Arezzo)
Tel. +39 055 96 21 11

Fax. +39 055 96 21 200
www.morettispa.com
info@morettispa.com