MANUALE D'USO

USER MANUAL

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MANUAL DE UTILIZACION

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MY STIM 2

EGREGIO CLIENTE

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ENGLISH

- Index	19
- Accessories	20
- Precautionary measures	21
- Palm technical features	23
-Electrode and battery removal	
- Description of the Palm	24
- Display	25
- How to use the Palm	26
-First use	
-How to pause the program	
-Automatic function	
-Automatic switching-off	
-Electrode contact	
-Empty battery	
-Memory function	
-Intensity of stimulation	
-Special functions	
-Keyboard lock key	
- Battery replacement	
- How to connect the cables	
- How to place the electrodes	
- Index of programs	29
- Common questions	33
- Guarantee terms	34
- Electrode Placement	83

Accessories

- Electro stimulator
- Carry case
- 2 connection cables
- Case of 4 reusable adhesive electrodes (5 x 5 cm)
- Case of 4 reusable adhesive electrodes (5 x 9 cm)
- 3 battery stylos 1,5 V AAA
- Operating manual

Accessories on request

- Ball pen
- Kit splitter (2 connectors + 2 connection cables) (4 channels with 8 electrodes)
- Conductive band for legs
- Face electrodes
- Y cables
- Gel

This stimulator is supplied complete of cables and electrodes. When you open the packing verify that the simulator equipment is complete. If some of the above mentioned accessories are missing, you should immediately contact the shopkeeper. Also check the stimulator and electrodes integrity.

Electrodes: The electrodes are usable by only one user they are self-adhesive and do not need gel. The electrode cable is female. They are branded CE complied with MDD 93/42/EEC requirements.

All the information can be subject to changes without notice.

The symbols mean that the apparatus is in accordance with medical CE requirements type BF.

PRECAUTIONARY MEASURES

This symbol warns you to read the operating instructions prior to use

Contraindications

- Do not stimulate the neck region (carotid sinus).
- Patients who have a demand type cardiac pacemaker.
- Stimulation should be not used on cancer patients (previous permission received by the oncologist).
- Do not stimulate the cerebral zone.
- Do not use on undiagnosed pains until aetiology is established.
- Do not use on sores and dermatological diseases.
- Do not use on acute traumas.
- Do not stimulate recent cicatrix.
- Do not use during pregnancy.
- Do not use driving car.
- It is absolutely forbidden to stimulate the ocular and genital zone
- It is also forbidden the use of the stimulator in the water (it is not protected against the water seepage). Not to use the stimulator in environment where combustible gas, inflammables, or oxygen are present.
- It is also required to use with caution in case of capillary fragility because an excessive stimulation could cause the break of more quantity of capillaries. Anyway it is always preferable to contact your doctor before using.

Adverse possible effects

Skin irritation can occur in rare case. In case of allergic reactions by the use of gel, stop the treatment and contact the specialist doctor.

If during the stimulation treatment tachycardia or extrasistolia should appear, it is better to stop the treatment the sensation will pass immediately.

Precautions for using

- Do not make any modifications or repairs by yourself. Authorized centres can only execute the repairs.
- Do not give a shock to the product by clashing, overturning or shaking. It may cause a trouble or accident in a long run although no immediate trouble occurs.
- Store the electro stimulator at a place free from a splash or water. Avoid installing at places subject to undue humidity, temperature, dust or places exposed to direct sunbeam.
- Do not use thinner, gasoline, kerosene, cleaning powder or chemicals; for cleaning the machine it is sufficient a soft duster
- Take away the battery from the electro stimulator in case of prolonged no use.
- After a long period of no use it is advisable to check the correct working.
- For cleaning the machine and the cables, it is sufficient a soft duster lightly humid.

Warnings

- The stimulator must be used only for transcutaneous neuro-stimulation.
- The neurostimulation apparatus should be used only under the continued supervision of a physician.
- The stimulator should be only used with skin electrodes intended for nerve and muscle stimulation.
- The user must keep the device out of reach of children.
- Electronic monitoring equipment (such as ECG monitors and ECG alarms) may not operate properly when stimulation is in use
- Stimulus delivered by this device may be sufficient to cause electrocution: Electrical current of this magnitude must not flow through the thorax because it may cause a cardiac arrhythmia.
- Do not use the apparatus in case of any health problem without consulting your doctor.
- In case of damage of the covering apparatus, this has to be replaced.
- It is advisable not to use the apparatus together with other electronic equipment above all those used for support of vital functions.
- The device must be used only by person with mental competence.
- Do not use the stimulator near short wave devices.

Use destination

Read the operating manual carefully in order to avoid to carry out installation, use and maintenance non adequate that could damage the apparatus the operator and the user.

The stimulators are used in the following operating environment and field:

- home;
- surgeries hospital;
- physiotherapeutic surgery;
- rehabilitation centres;
- treatments of pain in general
- in any type of sequential electro stimulation,
- Aesthetic and sports use,

The use of the stimulator is allowed to patients (rightly informed about the methods), doctors and paramedic.

The presence of clinic stimulation, makes this product a medical device.

The certification cover the clinic applications.

TECHNICAL FEATURES

Number of channels:	2.
Out current:	0 - 100 mA (peak to peak)
Wave form:	Square, biphasic, symmetrical, compensated impulse
Running frequency:	1-120Hz.
Impulse amplitude:	50-320 μ s.
Supply:	3 batteries of 1,5V AAA type LR03.
Storage and transportation temperature:	from -10°C to 45°C
Max. relative humidity:	75%
Atmospheric pressure:	from 700 hPa to 1060 hPa
Use conditions	
Temperature of use:	from 0°C to 40°C
Max. relative humidity:	from 30% to 75%
Atmospheric pressure:	from 700 hPa to 1060 hPa
Weight:	95 gr.
Classification:	Power supply type BF.
Certification:	Certificate CE MDD (Medical Device Directive)

The name of producer is reported in the last page.

Device removal

Don't throw the battery or electrodes out but recycle or get rid of them appropriately following the removal rules in force in your country.

DESCRIPTION OF THE PALM

ON / OFF key

- Press for three seconds for turning the stimulator ON.
- Press for three seconds for turning the stimulator OFF

LOCK key

- To lock all keys, except for the ON / OFF key, press for 3 seconds.
- To unlock the keys press for 3 seconds again.

Display

- Indicates mode, parameters etc.

PAUSE

- This switch can be used during the treatment as a stop button

Program key

- Press this key to select the program.

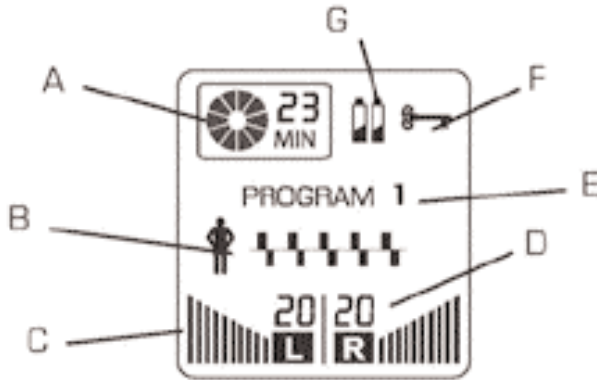
Intensity key of the left channel

- Press the key with the arrow ▲ to increase the out intensity of the left channel.
- Press the key with the arrow ▼ to decrease the out intensity of the left channel.

Intensity key of the right channel

- Press the key with the arrow ▲ to increase the out intensity of the right channel.
- Press the key with the arrow ▼ to decrease the out intensity of the right channel.

The device has a big display, which continuously gives information about the different parameters in use.

A Timer:

Shows the remaining time in minutes. The timer zone consists of a circular graph representation, which moves during the performance.

B Performance indicator:

The figures of small man and impulses, indicate the correct execution of the program. The represented graphic impulses are animated in different ways for each program. When this zone is light, it means that a program is in progress.

C Intensity graphic indicator:

The graphic bar shows the intensity level in use.

D Intensity numerical indicator:

Shows the used intensity level included from 0 to 40 scale.

E Program indicator:

Shows the selected program.

F Lock key indicator:

It shows that the keyboard is locked except for the ON/OFF switch.

G Battery indicator:

The indicator lightens when the batteries are almost empty.

HOW TO USE THE PALM

First use

1.  Attach the electrodes to the cable and place them on the skin.
(See Placement of Electrodes chapter of this manual).
2.  Turn the stimulator ON by pressing for three second the ON/OFF key. The stimulator automatically shows the last used program. The intensity indicator is positioned on 0 in both channels.
3.  Select the program you want to perform pressing the P+ or P- key.
4.  Increase the used channel intensity by pressing the arrow of (L) switches ▲ Increase the intensity until the wished level, maintaining a pleasant feeling.
To decrease the intensity press the arrow ▼.
5.  Increase the used channel intensity by pressing the arrow of (R) switches ▲ Increase the intensity until the wished level, maintaining a pleasant feeling.
To decrease the intensity press the arrow ▼.
6.  Every time you want to turn-off the stimulator, press the ON / OFF key. It can be used as an emergency button.

NOTE: Each program has a predefined duration time. The intensity resets to 0 when the program ends.

How to pause the program

You can pause the program by pressing the ON / OFF switch. Start the stimulation by pressing the On / OFF switch again. The program continues where the stimulation was shut off.

Automatic functions

Automatic Switching-off

If the electric stimulator is not in use for more than 5 minutes and the intensity in both channels is on 0, the stimulator automatically switches-off.

Electrode contact

If the electrodes are not correctly placed on the skin or the cables are not well connected, the intensity of channels is automatically shut off to 0.

In case the electrodes are not correctly placed before starting a treatment, it is not possible to increase the channel intensity.

This is a security measure to preserve the user from unpleasant possible effects.

Empty battery

The battery indicator lights up on the display when the batteries are almost empty. As long as stimulation feels the same as usual you can continue the treatment. It is time to replace the batteries when stimulation starts diminishing or the stimulator shuts off.

Memory function

The electric stimulator keeps on the memory the last performed program which is repurposed when you turn on the stimulator again. The memory function takes back the unit to the first program every time you change the batteries.

Intensity of stimulation

The intensity value for obtaining a special contraction is completely subjective. It can depend on a series of factors like as the electrode position, the adipose stratum, the perspiration etc. For these reasons the same current intensity value can give different sensations from a person to another from a day to another or furthermore you could feel more contraction on the left limb rather than the right one.

You have to regulate it on the bases of your sensations.

Key words: **Mod** = Moderated - **Elev** = Elevated - **Int** =Intermediate - **Max** = Maximum

Special Functions

Keyboard lock key

When you have reached the wished intensity level, you can lock the keys except for the ON / OFF switch, by pressing the LOCK key for 3 seconds. This function will avoid you the accidental use of switches during the treatment. The keyboard lock is released when the key is pressed for three seconds again.

Note: You cannot change the intensity value during the rump up and ramp down in all intermittent programs, which foresees cycles of work and recovery.

Battery replacement

Shut off the stimulator. Remove the battery cover by sliding it downwards and take out the batteries. Place 3 alkaline batteries of 1,5V AAA type LR03 and close the cover. Make sure that the new batteries are correctly placed with respect to the polarity (note marks for + and-)

Electrode and battery removal

Don't throw the batteries or electrodes out but recycle or get rid of them appropriately following the removal rules in force in your country.

How to connect the cables

It is possible to use one or two cables depending on use of one or two channels.

Connect the cable connector to the relative output placed on the inferior part of the unit. The outputs are exactly below the correspondent channel (left or right).

How to place the electrodes

Remove the electrodes from the original wrapping and connect the cable plugs to them. Place the electrodes on the skin following the images present in this manual. At the end of the treatment put back the electrodes in their special seat.

Note: The new electrodes have a hermetic wrapping, therefore check that they are new and supplied together with the unit.

ATTENTION: Don't remove the electrodes when the unit is working.

MY STIM2 - PROGRAM LIST SPORT

N° Program	Program number	Electrode placement	Phase time (min.) Intensity
1	DEMO	As indicated in the figures at the end of the manual	2' ☐ 2' ☐ 2' ☐ 2' ☐ Mod Mod Mod Mod
2	Search motor points	As indicated in the figures at the end of the manual	2' ☐ ☐ ☐ Mod
3	Capillarisation	As indicated in the figures at the end of the manual	20' ☐ ☐ ☐ Mod
4	Warm Up	As indicated in the figures at the end of the manual	10' ☐ ☐ ☐ Mod
5	Tension relieving	As indicated in the figures at the end of the manual	20' ☐ ☐ ☐ Mod
6	Post competition/ workout recovery	As indicated in the figures at the end of the manual	20' ☐ ☐ ☐ Mod
7	Maximal power - lower limbs	I1-I2-I3-I4-I5-I6	5' ☐ 15' ☐ 5' ☐ Int Max Int
8	Resistance - lower limbs	I1-I2-I3-I4-I5-I6	5' ☐ 20' ☐ 5' ☐ Int Max Int
9	Explosive power - lower limbs	I1-I2-I3-I4-I5-I6	5' ☐ 12' ☐ 5' ☐ Int Max Int
10	Maximal power - torso and upper limbs	S1-S2-S3-S4-S5-S6-S7-S8	5' ☐ 15' ☐ 5' ☐ Int Max Int
11	Resistance - torso and upper limbs	S1-S2-S3-S4-S5-S6-S7-S8	5' ☐ 12' ☐ 5' ☐ Int Max Int
12	Explosive power - torso and upper limbs	S1-S2-S3-S4-S5-S6-S7-S8	5' ☐ 12' ☐ 5' ☐ Int Max Int

MY STIM2 - PROGRAM LIST *FITNESS*

N° Program	Program number	Electrode placement	Phase time (min.) Intensity
13	Relaxing massage	As indicated in the figures at the end of the manual	25' Mod ☐ ☐ ☐
14	Toning massage	As indicated in the figures at the end of the manual	20' Mod ☐ ☐ ☐
15	Vascularisation - lower limbs	I1-I2-I3-I4-I5-I6	25' Int ☐ ☐ ☐
16	Drainage - lower limbs	I1-I2-I3-I4-I5-I6	30' Int ☐ ☐ ☐
17	Lipolysis - lower limbs	I1-I2-I3-I4-I5-I6	40' Int ☐ ☐ ☐
18	Firming - lower limbs	I1-I2-I3-I4-I5-I6	20' 10' Ele Int ☐ ☐
19	Toning - lower limbs	I1-I2-I3-I4-I5-I6	20' 10' Ele Int ☐ ☐
20	Modelling - sculpting - lower limbs	I1-I2-I3-I4-I5-I6	20' 20' Max Int ☐ ☐
21	Increasing mass - lower limbs	I1-I2-I3-I4-I5-I6	15' Max ☐ ☐ ☐
22	Drainage - abdominal belt	T1-T2-T3-T4-T5-T6	30' Int ☐ ☐ ☐
23	Lipolysis - abdominal belt	T1-T2-T3-T4-T5-T6	40' Int ☐ ☐ ☐
24	Firming - abdominal belt	T1-T2-T3-T4-T5-T6	20' 10' Int Int. ☐ ☐
25	Toning - abdominal belt	T1-T2-T3-T4-T5-T6	20' 10' Int Int ☐ ☐
26	Modelling - abdominal belt	T1-T2-T3-T4-T5-T6	20' 20' Int Int ☐ ☐
27	Drainage - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	30' Int ☐ ☐ ☐
28	Lipolysis - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	40' Int ☐ ☐ ☐
29	Firming - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	20' 10' Int Int ☐ ☐
30	Toning - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	20' 10' Ele Int ☐ ☐
31	Modelling - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	20' 20' Max Int ☐ ☐
32	Increasing mass - upper limbs and pecs	S1-S2-S3-S4-S5-S6-S7-S8	15' Max ☐ ☐ ☐

MY STIM2 - PROGRAM LIST *FITNESS*

N° Program	Program number	Electrode placement	Phase time (min.) Intensity
33	Drainage - thighs and hips	I1-I2-I3-I4-I5	30' ■ Int
34	Lipolysis - thighs and hips	I1-I2-I3-I4-I5	40' ■ Int
35	Firming - thighs and hips	I1-I2-I3-I4-I5	20' 10' ■ Ele ■ Int
36	Toning - thighs and hips	I1-I2-I3-I4-I5	20' 10' ■ Ele ■ Int
37	Drainage - gluteus	I6	30' ■ Int
38	Lipolysis - gluteus	I6	40' ■ Int
39	Firming - gluteus	I6	20' 10' ■ Ele ■ Int
40	Toning - gluteus	I6	20' 10' ■ Ele ■ Int
41	Modelling - gluteus	I6	20' 20' ■ Ele ■ Int
42	Firming - breast	S9	15' ■ Mod
43	Modelling - breast	S9	10' ■ Mod
44	Capillarisation - face	V1	15' ■ Mod
45	Face lifting effect	V1	15' ■ Mod

MY STIM2 - PROGRAM LIST *HEALTH and REHAB*

N° Program	Program number	Electrode placement	Phase time (min.) Intensity
46	Analgesic effect by tens for acute pain	As indicated in the figures at the end of the manual	30' Mod
47	Analgesic effect by tens for chronic pain	As indicated in the figures at the end of the manual	30' Mod
48	Endorphin effect by tens	As indicated in the figures at the end of the manual	30' Mod
49	Hematoma	As indicated in the figures at the end of the manual	20' Mod
50	Sciatica	R7	30' Mod
51	Recovery from atrophy - lower limbs	I1-I2-I4	5' 15' Int Ele
52	Recovery from atrophy - upper limbs	S1-S2-S3-S4-S5-S6-S7-S8	5' 15' Int Ele
53	Backache	T3	20' 20' Mod Mod
54	Cervical pain	R1	30' Mod
55	Periarthritis	R2	15' 10' Mod Mod

* Programs and protocols of health and tens, are generic suggestion about the treatments to be executed. They must be confirmed by a doctor or specialist.

COMMON QUESTIONS

Why does the intensity return immediately to 0 when increasing its value?

The unit has a control of current supply. In case the circuit is open the intensity is automatically shut off. It is therefore necessary, before starting any treatment, to be sure that the cables are well connected and the electrodes completely placed on the skin.

What to do when the discharged battery indicator appears?

You can continue your stimulation until it starts diminishing; then turn off the unit, disconnect the electrode cables and change the batteries.

Is it possible to use the electric stimulator during the menstrual cycle?

Possible interferences, like anticipations, delays, accentuations or reductions of the cycle can be subjective and variable. It is however advisable to avoid treatments in the abdominal zone during the cycle.

Is it possible to use the electric stimulator during lactation?

Until now it has not found collateral effects in relation to the electric stimulator use during the lactation. It is however advisable not to treat the thoracic zone.

Does the presence of dermatological diseases like psoriasis, urticaria, etc. prevent the electric stimulator use?

Yes, it is advisable not to treat epidermic zones, which present important dermatological diseases.

How long after is it possible to see the first results?

The aesthetic results obtained are always subjective. It is however possible to state that you can achieve good results about tonicity after 15 days with 3 or 4 sessions a week; for lypolysis and drainage treatment it is necessary at least 40 days. Better and speedier results are obtained if treatments are combined with a good physical activity and a correct life style.

GUARANTEE TERMS

This product is warranted to the first user for a 24-month period from the purchase date against material defects or manufacturing, provided that it is used properly and maintained in the normal efficiency conditions.

To enjoy the warranty service, the user must observe the following warranty articles:

1-The products must be delivered for repairing by the User in their original packaging, the expenses are charged to the User.

2-The product warranty depends on the presentation of fiscal document (cash voucher, fiscal receipt or selling invoice) and certifying the purchase date of the product.

3-The repairing will not have effect on the original date of warranty expiration and will not mean the renewing or extension of the warranty.

4-When during the repairing will not be found any defect, will be in any case charged the expenses related to the checking time.

5-The warranty doesn't have effect in case of damages by bits, falls, improper or incorrect use, not original external supply, accidents, alterations, changes / removal of warranty seals and / or product infringement. The warranty doesn't cover the damages caused during the transportation due to the use of inadequate boxing (see art. 1).

6-The warranty doesn't cover the impossibility of using the product, other occasional or consequential expenses or other purchaser expenses.

ATTENTION: Before returning the machine to be repaired, it is recommended to carefully read the user manual instructions. In case of device breakdowns contact the shopkeeper before. The fabricant can make at any moment all the necessary modifications to improve the product aesthetic and quality.

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This product as been manufactured by: SAVIA L.t.d. - 2nd Industrial District, Feng Huang Village (Shenzen) - Made in PRC - sales@tyece.com

Agency: Domino s.r.l. - via San Felice, 4 - 31020 San Vendemiano - TV -ITALY - info@domino.tv.it

Distributed by: Globus Italia s.r.l. - via Vittorio Veneto, 52 - 31013 Codognè -TV -ITALY - Tel. (+39) 0438.7933 - info@globusitalia.com

This product has been manufactured according to the technical regulations in force and it has been subjected to certifications according to 93/42/CEE directive for medical devices in order to assure the product's security. The user must follow all prescriptions and restrictions shown in the operating manual in order to keep the maximum security level of the product. The manufacturer declines all responsibility for every different use from what indicated in this operating manual. All complete or partial reproductions of texts and/or pictures of this operating manuals made by any electronic or mechanic equipments are strictly forbidden without written permission of the manufacturer.

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