Nocturno®

ELECTROMAGNETOTERAPIA DE ALTA FRECUENCIA
TERAPIA ELECTROMAGNÉTICA DE ALTA FREQUÊNCIA
ELETTROMAGNETOTERAPIA ALTA FREQUENZA
HIGH FREQUENCY ELECTROMAGNETIC THERAPY
ÉLECTRO-MAGNÉTHÉRAPIE HAUTE FRÉQUENCE

E

PT

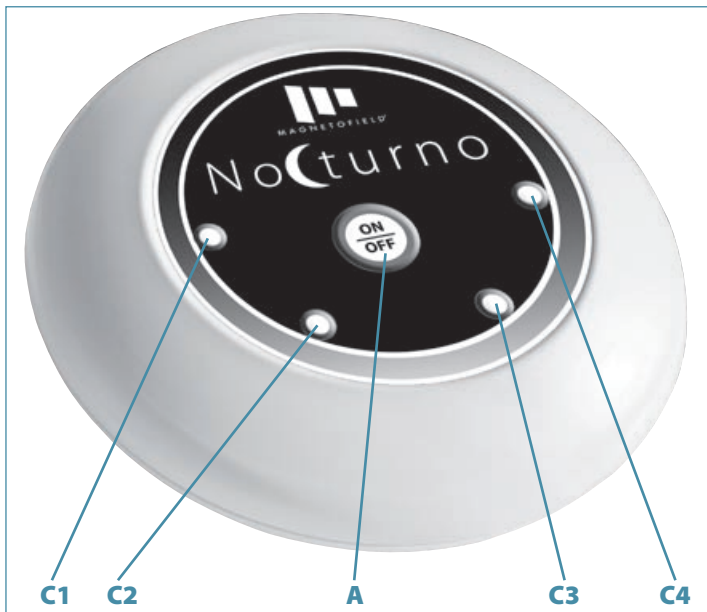
IT

UK

FR

MANUAL DE USO • MANUAL DO USUÁRIO • MANUALE D'USO • USER MANUAL • MANUEL D'UTILISATION

**A**

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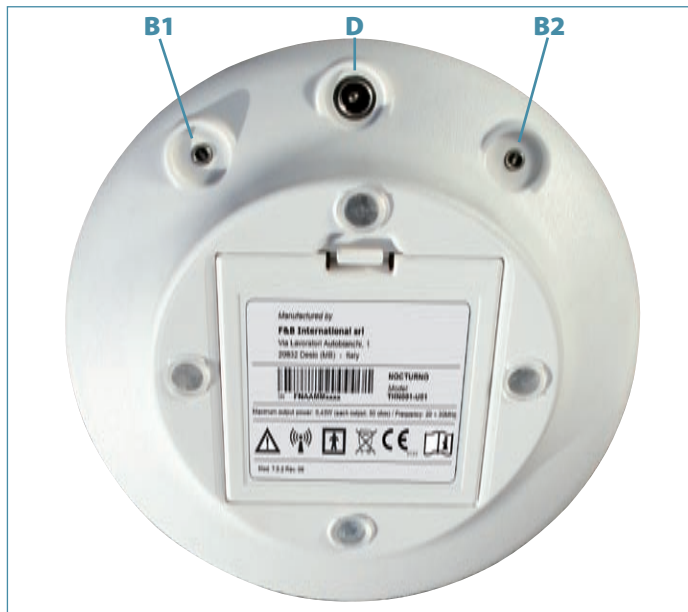
*Electromagnetotherapy outputs
to connect accessories*

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*Connection for battery-charger
power unit*

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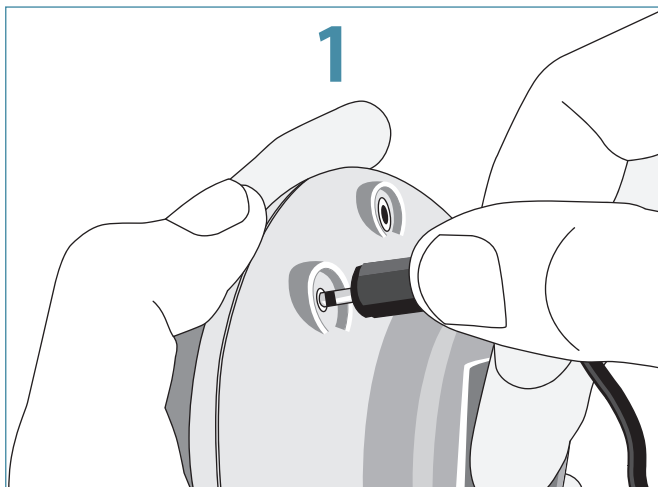
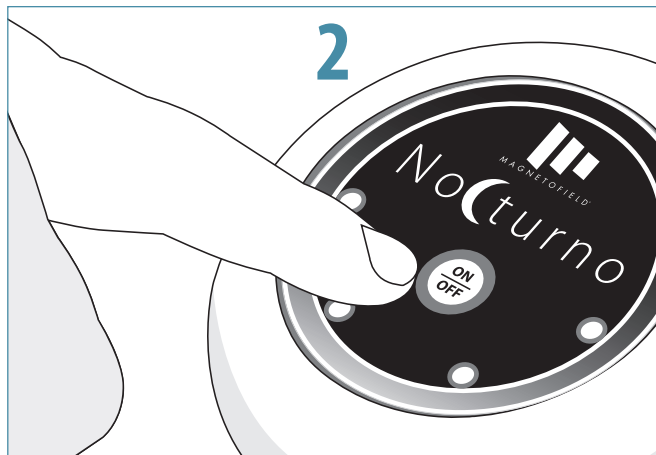
Press the ON button

The Led (C1, C2, C3, C4) will light up in blue to indicate that the therapy is started.

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The pictures that follow are merely indicative and not binding of the company.

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Textile accessory multifunction (23x 43 cm)

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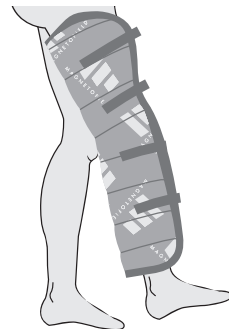
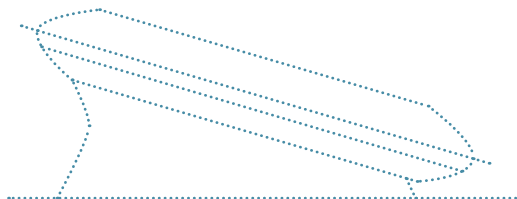
NOCTURNO is registered with the Ministry of Health as a Class IIA Medical Device with n. 802089.

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Product Service

EC Certificate

Full Quality Assurance System
Directive 93/42/EEC on Medical Devices (MDD), Annex II excluding (4)
(Devices in Class IIIa, IV or V)

No. G1 13 04 84243 001

Manufacturer:	F & B INTERNATIONAL S.r.l. Via Lavoratori Autobianchi, 1 – PTB 14N 20832 DESIO (MI) ITALY
Facility(ies):	F & B INTERNATIONAL S.r.l. Via Lavoratori Autobianchi, 1 – PTB 14N, 20832 DESIO (MI), ITALY
Product Category(ies):	Electromagnetic stimulation devices and relative accessories

The Certification Body of TÜV SÜD Product Service GmbH declares that the aforementioned manufacturer has implemented a quality assurance system for design, manufacture and final inspection of the respective devices / device categories in accordance with MDD Annex II. This quality assurance system conforms to the requirements of this Directive and is subject to periodic surveillance. For marketing of class IV devices an additional Annex II (4) certificate is mandatory. See also notes overleaf.

Report No.:	ITA 231911
Valid from:	2013-04-05
Valid until:	2018-04-04

Date: 2013-04-08
Hans-Henner Junke

TÜV SÜD Product Service GmbH is Notified Body with identification no. 0123

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CERTIFICATE

No. Q1N 13 04 84243 002

Holder of Certificate:	F & B INTERNATIONAL S.r.l. Via Lavoratori Autobianchi, 1 – PTB 14N 20832 DESIO (MI) ITALY
Facility(ies):	F & B INTERNATIONAL S.r.l. Via Lavoratori Autobianchi, 1 – PTB 14N, 20832 DESIO (MI), ITALY
Certification Mark:	
Scope of Certificate:	Design and development, production and servicing of electromagnetic stimulation devices and relative accessories
Applied Standard(s):	EN ISO 13485:2012 / AC: 2012 Medical Devices - Quality Management Systems - Requirements for regulatory purposes

The Certification Body of TÜV SÜD Product Service GmbH certifies that the company mentioned above has established and is maintaining a quality system which meets the requirements of the listed standard(s). See also notes overleaf.

Report No.:	ITA 231911
Valid from:	2013-04-05
Valid until:	2018-04-04

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
TIMBRO RIVENDITORE

CONTENTS

Electromagnetotherapy (EMT)	<i>page</i>	43
Therapeutic indications	<i>page</i>	44
Safety measures - Maintenance	<i>page</i>	44
Replacing batteries	<i>page</i>	45
Contraindications and precautions	<i>page</i>	45
Meaning of the label and symbols	<i>page</i>	46
Technical specifications	<i>page</i>	47
How to Use NOCTURNO®	<i>page</i>	48
Batteries and recharge	<i>page</i>	48
How to best use NOCTURNO®	<i>page</i>	49
Applications and Index of Pathologies	<i>page</i>	50
Warranty terms and Certificate	<i>page</i>	52

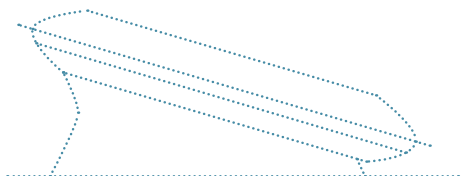Dear and Valued Customer,

for years we have been developing natural therapies that contribute to improve human life, diligently following the European Directives that concern Medical Devices.

We pay the utmost attention and care to every detail, technological and functional, to offer equipment that fully answer to therapeutic and wellness needs, combining maximum safety with the best comfort.

In sincerely thanking you, we recommend that you carefully read this Manual, to know how to best use our product and thus obtain the best benefits.

The observations and suggestions you will provide will make it possible to put your satisfaction to the service of others.

This manual must be stored, together with the relative Medical Device, for it to be consulted whenever necessary.

More specifically, in case the device is sold or transferred to another user, make sure that the Manual "follows" the equipment, to guarantee that the new user may be thoroughly informed concerning the most appropriate use.

ELECTROMAGNETOTHERAPY (EMT)

A BRIEF HISTORY

The first observations on the interactions of magnetic fields on human health have very ancient origins, and can be traced as far back as Hippocrates and the ancient Egyptians.

The first systematic studies and the observations on the relation between magnetic fields and living organisms, instead, first started toward the end of the 16th century.

But it is only with the scientific discoveries and the technological progress made in the last century that the extraordinary drive to knowledge and to the possible applications of electromagnetotherapy in Medicine was set into motion, especially thanks to Russian, American, Japanese and French studies (Danielewsky, Basset, Pilla, Fukuda, Yasuda, D'Arsonval, Fellus, etc.).

For these reasons, the validity of the application of magnetic fields in the therapeutic field was only officially recognized in the medical field in the last decades.

APPLICATIONS AND BENEFITS

We will discuss:

High Frequency, for frequencies in the realm of radio-waves, between 1 and 300 MHz with carrier frequency between 20 and 30 MHz with relative harmonics, pulsed and at extremely low power, and operating frequencies up to 5,000 Hz).

The High Frequency (HF) impulses improve blood circulation and are capable of stimulating the production of endorphins by the neuro-vegetative system, with

the consequential reduction of the pain associated to the various pathogenic statuses and an effective anti-inflammatory action.

For these reasons, both acute and chronic pathologies can be treated, ensuring a curative and/or improving action on the condition of health of the patient.

The benefits of magnetotherapy are therefore multiple; we list some of them below:

- **Anti-inflammatory effect (HF):**
- **Antalgic/pain relief effect (HF):**
 - stimulates the production of endorphins;
- **Effects on bone tissue (HF):**
 - increase of bone resistance;
 - improvement of osteogenesis;
 - increase of mineralization;
 - accelerate the calcification in the fractures;
- **Blood vessel active effect (HF):**
 - increase of peripheral hematic flow and of the speed of blood flow;
 - increase of vascular blood flow;
 - improvement of oxygenation and reduction of liquid accumulation;
- **Anti-edema action and tissue repair (HF):**
 - accelerates the healing of wounds/scars;
 - acceleration of the healing processes of soft tissues;
- **Bacteriostatic/anti-allergic action (HF);**
- **Neuro-balancing action (neuro-adjustment on the hypothalamus, the liver, and the spleen) (HF);**
- **Improvement of the skin and anti-ageing action on tissues (HF).**

In summary, the beneficial outcomes of the use of

Pulsed ElectroMagnetic Fields (PEMF) are especially obtained on an articular level (*arthritis, arthrosis, osteoporosis, bone fractures, etc.*), on a muscular/tendon level (*lumbago sciatica, tendonitis, sprains, rheumatic pains, etc.*), on a vascular level (*phlebitis, varicose veins, edemas, bedsores, healing of wounds, etc.*), and with problems tied to stress (*sleep disorders, anxiety, etc.*).

OUR EQUIPMENT: NOCTURNO®

The fruit of many years of research and experience, as well as of constant clinical experimentation, the Electromagnetotherapy of our devices gathers into it the very best of the most advanced technology and the ambition to guarantee the best therapeutic results, along with the utmost easiness of use.

NOCTURNO® works with High Frequency (operating frequencies from 100 to 5,000 Hz.) for therapies to be performed lying down and preferably at night.

The treatment is managed by a microprocessor and works with automatic scan: in other words, it is sufficient to start the therapy and the equipment directly emits the consequential electromagnetic field.

Moreover, it works without the need of being connected to an electrical network, except when it is necessary to recharge the internal batteries.

All of these technical features have been realized to guarantee its use within the home of the end user: in order to enjoy a device, safe and guaranteed, comfortably and easily within one's home.

THERAPEUTIC INDICATIONS

NOCTURNO® is especially suited for:

- ✓ INFLAMMATORY PAINS - NEURALGIA
- ✓ ARTICULAR PAINS
- ✓ POST-TRAUMATIC PAINFUL CONDITIONS
- ✓ CERVICALGIA AND HEADACHES
- ✓ PERIPHERAL CIRCULATION DISORDERS AND EDEMAS
- ✓ PROBLEMS TIED TO STRESS, SLEEP DISORDERS, AND NERVOUS TENSION

SAFETY MEASURES - MAINTENANCE

The Medical Device is manufactured in accordance with the Directives and Norms in force for Medical Devices. Since the device is powered by batteries (the external power unit can be used only for recharging the batteries), there are no special precautions to observe when using it.

We recommend that you do not store the device with fully dead batteries.

The batteries supplied last long and are without memory effect, which means they can be recharged by means of the adapter supplied, in any moment (without having to remove them from the compartment in which they are placed), even if they are not fully dead.

For more detailed information on the batteries and on how to replace them, carefully read pages 45 and 48.

Please be reminded that the batteries and the whole device must be disposed of separately when no longer functional, in accordance with the norms of law in force.

For safety and reliability purposes, exclusively use the battery charger supplied with the device.

For any orders you may need to make, see the "Spare parts" paragraph.

CAUTIONS

The device is guaranteed, provided that:

- The repairs are carried out by authorized personnel or at our company premises;
- The electrical system in the work environment is compliant with the safety norms in force;

- The identification label that bears the model, the Serial Number (SN), and the bar code, is intact (the same holds true for the labels with the bar codes of the accessories); any attempt to open the device entails termination of the warranty;
- All the instructions contained in this manual are followed.

An improper use of the device and/or battery entails termination of the warranty.

SPARE PARTS

The manufacturing company makes the spare parts of the accessories supplied with the device available. Therefore, for future orders, directly contact the company or the authorized Retailers.

WARRANTY

The device and its accessories are covered by a warranty of 24 months for manufacturing flaws. The warranty does not include the aesthetic parts and the consumption material that is not prone to wear-out (except for abnormal conditions of the manufacture), an also any damages/faults caused by a clumsy/improper use of the machine and/or the accessories.

Warning:

Failure to present the warranty certificate implies its termination, if interventions were to prove necessary on the device, they will be responsibility of the user.

CLEANING

The Device can be cleaned with a normal soft and dry cloth. More die-hard stains can be removed by using a cloth slightly dampened with an alcohol solution. For its disinfection, treat the device with the protective solutions normally used for electro-medical devices.

ORDINARY MAINTENANCE

The manufacturer deems itself responsible for the reliability and safety of the device, only if:

- Possible modifications or repairs are carried out by explicitly authorized personnel;
- The electrical system of the work environment where the Device is connected to recharge the batteries is compliant with the Safety Norms in force;
- The device is used in strict compliance with all the instructions for use and the warnings contained in this Manual.

FIRST INTERVENTION

In the case that the device fails to operate, we recommend that you perform the following operations (before contacting Assistance):

- Check that all the instructions for use have been followed properly, as suggested in the Manual;
- Check that the accessories and the wires are in good contact and condition;
- Charge the batteries with the specific charger supplied.

TECHNICAL ASSISTANCE

For any possible request for technical assistance, please contact our office or our authorized centers.

REPLACING BATTERIES

Operations to be performed with the device off and not connected to the network.

1. Place the device on top of a table, laid on a cloth, with its bottom facing upward.
2. Open the compartment where the batteries are lodged.
3. Pull out the batteries: the 4 batteries must be replaced simultaneously; the partial replacement of the batteries is forbidden.
4. Insert the new batteries: comply with the polarity and battery type (Ni-MH batteries, mini-stylus size AAA, rechargeable, 1.2 V 700/800 mAh).

For safety reasons, it is forbidden to use non-rechargeable batteries: they could get damaged if recharged.

5. Close the battery compartment again.

Please be reminded that batteries that no longer work must be disposed of separately, in compliance with the laws in force.

CONTRAINDICATIONS AND PRECAUTIONS

Based on the current level of medical knowledge, there are **3 CONTRAINDICATIONS** to the use of electromagnetotherapy devices. Nonetheless, it best to observe certain precautions, both with regard to the patients subjected to the therapy and with regard to the proper use of the device.

- **Persons with pacemakers must not be subjected to instrumental therapies** except if with precise medical indication and under close surveillance;
- **In cases of pregnancy or hemophilia, only a doctor's opinion can recommend the therapy** which must at any rate be carried out under close medical control;
- **For patients who manifest serious pathologies on a cardiologic or neurological level,** considering the delicate nature of the cases in question, we recommend at any rate that **you first consult a doctor** before carrying out the electromagnetic therapy.

PRECAUTIONS

- After their purchase, make sure that the equipment and the accessories supplied are intact: in case of doubt, do not use them and contact the retailer or manufacturer;
- More specifically, check the labels with the bar codes: please be reminded that removing the labels will entail termination of the warranty;
- The electromagnetotherapy can be used (since it is a wave and not a current) in the presence of

wounds, ulcers, edemas, by interposing sterile gauze between the accessory and skin;

- Children or at any rate minors are forbidden to make direct use of the electro-medical devices, unless they are assisted by responsible adults;

- It is normal that during the first treatments the patient may perceive a light increase of the pain sensation, especially if the field emitted goes to contrast an area where inflammatory processes are occurring. If the painful sensation is bothersome and unbearable, we recommend that you temporarily interrupt the therapy (for 3/5 days), to then resume it and finish it;

- It is at any rate a good practice to consult your doctor and have him advise you on the best use of the equipment or to clarify any doubts concerning the treatments;

- Only use the accessories supplied with device** (cables, charger, etc. indicated in this manual) or supplied by the manufacturer/assistance: **it is forbidden (for safety reasons) to use any other type of accessory; it is likewise forbidden to use the accessories supplied with the device on other equipment.**

- The accessories that are directly applied on the skin are to be used on one patient only; if otherwise, they must be washed and cleaned to guarantee maximum hygiene safety and to prevent possible infections/contaminations;

- We recommend that you periodically check the condition of the accessories, checking, in particular, that the wires and the parts applied are not damaged;

- The device and its relative accessories must not be exposed to sources of heat;

- Use in contact with water is forbidden;

- Do not use the Device in the immediate vicinity of other equipment (in particular at short or micro-wave), and do not connect them to the same power socket (this could compromise their proper operation);**

- Do not simultaneously use the device with an HF surgical device connected to the patient;

- Treatments with Electromagnetotherapy do not contraindicate, at any rate, the use of other prescribed therapies (manual, instrumental, or pharmaceutical);

- It is severely forbidden to open the device. This operation is only allowed to an assistance center authorized by the manufacturer;; no modification to the device is allowed;**

- It is at any rate explicitly forbidden and dangerous to make any use of the device other than the one indicated in this manual; the manufacturer cannot be deemed responsible for any damage caused by an incorrect, unreasonable improper use of the device;**

- Recommended environmental operating conditions:

Room temperature	⇒	15 ÷ 35 °C
Relative humidity	⇒	45 ÷ 75%
Atmospheric pressure	⇒	860 ÷ 1060 hPa
- Recommended environmental conditions for storage:

Room temperature	⇒	5 ÷ 50 °C
Relative humidity	⇒	30 ÷ 90%
Atmospheric pressure	⇒	860 ÷ 1060 hPa
- Recommended environmental conditions for transport:

Room temperature	⇒	0 ÷ 50 °C
------------------	---	-----------

MEDICAL DEVICE LABEL

(The SN and the bar code univocally identify the device and its date of manufacture)

SERIAL NUMBER (SN):

FN = Medical Device code

AA = last 2 digits of the year of manufacture

MM = last 2 digits of the month of manufacture

xxxx = progressive 4-digit number

MEANING OF THE SYMBOLS

Carefully read the contraindications and precautions.

Device that emits non-ionizing radiations.

Device with BF-type parts applied to the body.

Marking symbol (0123 is the identification number of the TÜV SÜD PS Certification Institute).

Device and battery subjected to separate disposal.

Carefully read the operating manual.

TECHNICAL SPECIFICATIONS

- IIA class Medical Device (Directive 93/42/EEC and 2007/47/EC) designed and manufactured for home treatments with high frequency ElectroMagneToTherapy (EMT).
- Desktop device, portable. Ø 12 – 5,5h cm
- Keypad in polycarbonate with on/off key.
- 2 outputs for EMT.
- Short wave therapy (EMT): [tolerance $\pm$ 30%]
High frequency
 Type of modulation: signals pack modulated with on-off with variable duty-cycle.
 Fundamental oscillation frequency: 20 ÷ 30 MHz
 Oscillation frequency: 100 ÷ 5.000 Hz
 Complete scan duration: 65 s.
- Powered with internal batteries 4.8 V – 700/800 mAh, Ni-MH RECHARGEABLE (4 mini-stylus AAA 1.2V).
- Power unit 100÷240V 12VDC 500-2500mA to recharge the batteries. [Medical Power Unit compliant with norm EN60601-1]
- Max. power in output: 0.43 W. (single output; effective value)
 (with non-inductive resistive load 50 Ω)
- Power reserve: Approx. 24 hours with maximum charge.
- Batteries recharge time: Approx. 4/5 hours.

ACCESSORIES SUPPLIED WITH NOCTURNO® (model THN001-U01)

- 1 external power unit to recharge batteries. (Code ALI004) *Image on page III*
- 1 cable to connect textile accessories (1.5 m) (Code CAV005) *Image on page III*
- 1 cable to connect textile accessories (2.5 m) (Code CAV005-L) *Image on page III*

OPTIONAL ACCESSORIES

- 1 textile accessory multifunction (23x43 cm). (Code FAS005-N) *Image on page III*
- 1 leg band. (Code GAM001-N) *Image on page III*
- Magnetic mat for bed (several sizes). (Code STU002-N) *Image on page III*

WASHING INSTRUCTIONS FOR TEXTILE PRODUCTS

DRY CLEAN ONLY WITH FOAM

Description	Symbol
Do not dry clean	
Do not wring; do not spin dry	
Do not iron	
Do not bleach	
Do not dry in machine dryer	
Do not wash in water	

HOW TO USE NOCTURNO®

Press the button "A".

The LED (C1, C2, C3, C4) will switch on to a green colour to indicate battery status.

The number of LED that are on informs the operator of the battery level:

- 4 LED on ⇨ approx. 100% charged;
- 3 LED on ⇨ approx. 75% charged;
- 2 LED on ⇨ approx. 50% charged;
- 1 LED on ⇨ approx. 25% charged.

The treatment starts immediately after, for a preset time of 6 hours ("UNIVERSAL" program).

The accessories must be connected to any one of the 2 outputs "B1" or "B2".

The total-body mat is especially suited for generalized problems which involve multiple body parts: perform the treatment lying on it.

During the therapy, the LED (blue color) alternatively scroll forward and backward to indicate that the cycle is being performed.

The "UNIVERSAL" program covers the entire range of frequencies available for the High Frequency: it therefore can be used with various pathologies, both acute and chronic; this program is set on **CONTINUOUS TIME of 6 hours**, to perform prolonged treatments, usually during night rest for comfort purposes.

At the end of the preset time (6 hours), the device automatically switches off.

If, instead, you wish to interrupt the treatment at any given moment, all you have to do is keep button "A" pressed for a long time (approximately 3 seconds): the treatment stops and all the LED switch off.

In addition to the switching on the device (see the

start of this page), to check battery charge status during the therapy, it is sufficient to briefly press (approx. 1 sec) the "ON/OFF key: the green LED will be displayed with the same percentage modes described previously. This display will remain for approximately 5 seconds, and then the status returns to the previous one (alternate blue LED).

It is important to remember that, once you have finished using **NOCTURNO®**, all the accessories must be disconnected from the device power sockets and stored away separately.

We recommend that you perform this operation by pulling the connector and not the cables, because they would otherwise get damaged in a short time.

The ElectroMagnetoTherapy (EMT) does not cause particular sensations, because it is a wave and not a current.

For this reason, it has a greater ability to penetrate than other instrumental treatments, with consequential therapeutic advantages.

It is normal that during the first treatments, the patient may perceive a light increase in the painful sensation, especially if the field goes to contrast an area where inflammatory processes are occurring. If the painful sensation is bothersome and unbearable, we recommend that you temporarily interrupt the therapy (for a few days), to then resume it and finish it.

BATTERY AND RECHARGE

We once again emphasize that, after the purchase, before the first use of NOCTURNO®, the device must be charged: insert (with the device off) the power unit into the electrical socket and then connect it to the machine in output "D".

When recharging, the LED "C1", "C2", "C3" and "C4" flash together in green and, once the charging is over (approx 4/5 hours), after a brief, faster flashing sequence, they switch off as also the device.

During standard use, the instructions on how to recharge the battery are pretty much the same to those described above: use, *with the device off*, the specific external power unit supplied, to be connected to the output "D" and to the network, until the green LED switch off and the device automatically stops.

Once you have finished recharging the battery, disconnect the external adapter: the device is now ready to perform new treatments

Only the power unit supplied with the device must be used.

During periods of device disuse, we recommend that:

1. Before putting the NOCTURNO® away for it to be stored, you recharge the batteries.
2. The batteries must at any rate be recharged at least once every 3 months
3. Put the device to recharge when you decide to reuse NOCTURNO®.

Failure to follow these instructions compromises device functionality and terminates validity of the warranty.

With the therapy in progress (i.e. device on with scrolling blue LED), if you insert the power unit to recharge the batteries, the treatment is interrupted, the blue LED switch off and they begin to flash together in green for the normal recharging mode. In this phase, the “ON/OFF” button (“A”) is disabled.

If during the recharging phase, the power unit is removed from output “D”, the device switches off. You will have to press the key “ON/OFF” to start a new treatment.

The batteries have a power reserve of at least 24 hours at full charge (in other words, with the continuous use of the accessories).

HOW TO BEST USE NOCTURNO®

The applications are preset to last 6 hours, usually at night, once a day, and for an overall variable length between 45 and 60 days.

This device, unlike our other Medical Devices, is supplied with a single program, which covers the entire range of useful frequencies.

The treatment periods can be periodically repeated (approximately 3 times a year) with an interval of at least 20/30 days, in order to optimize the therapeutic results.

You should always bear well in mind all the parameters that concur to optimizing the therapy: a check on the “receptivity” of the user toward the therapy and the observance of the precautionary norms.

The therapy with electromagnetic fields emitted by **NOCTURNO®** has a high therapeutic effectiveness and does not entail contraindications, except for persons with pace-makers, pregnant women, and in very severe cardiologic/neurological cases.

At the present state of knowledge, no relevant side effects have been detected.

It is nonetheless correct to specify that the way patients respond to the therapy is subjective, both in terms of results and in the time they take to improve/heal, which depend, in addition to the actual pathologies they are affected by, also on the way they interact with other physical or pharmaceutical treatments, on the psychological-physical condition of the user, on individual sensitivity to the therapy, on the outside environment, and on the proper use of the device and its accessories.

In view of these considerations, it is possible that not all the patients respond in the same way to the treatments, and certain general precautions must be held in account:

- Sleepiness may occur during treatments with magnetotherapy.
- At first, it is possible to encounter (especially in more complex pathologies, acute or chronic, e.g. arthritis, osteoporosis, etc.) periods of improvement that alternate with the intervals of “stagnation” or of slight worsening of the symptoms: continue with the therapy, because over time the organism will assimilate the benefits of the electromagnetic fields;
- It is normal that, in inflammatory or post-traumatic pathologies (tendinitis, cervicalgia, arthrosis, sprains, neuralgia, etc.) the painful sensation felt by the patient may at first increase;
- In extremely rare cases, the onset of a slight condition of hyperactivity, of insomnia, and/or of itching is possible (e.g. with psoriasis and with neuralgia), especially in patients who prove particularly sensitive to the treatment.

In these last two cases, it is advisable to temporarily suspend the treatments (a few days give or take), to then gradually resume the therapy.

The equipment is marked with the Serial Number (SN) and its relative bar code (see page 46); and the accessories too (wires for textile accessories, power unit, manual, etc.), as also its inner parts (circuit, etc.), are identified by a label with serial number and bar code.

There is a univocal association between the SN of the Medical Device and the bar codes of all the parts it comprises.

For this reason, the accessories must “follow” the equipment to which they are associated.

In the following part of the Manual (“APPLICATIONS” section):

- a. Certain pathologies that are treatable with this device are summarized and listed, with the recommendable relative accessories;
- b. All the information and the drawings reported below must be understood as indications, suggestions for the therapy: they do not replace, nor contrast, any possible instructions received by specialized medical/health staff.

APPLICATIONS

ANALYTICAL INDEX OF PATHOLOGIES

(The bibliography and more information about certain references on studies carried out with our Medical Devices and other equipment of magnetotherapy, are available and can be obtained from the company)

Pathologies / Dysfunctions	Group Reference (page 51)	Recommended accessories (page III)
ARTHRITIS	2	F-G-H
ARTHROSIS (CERVICAL - LUMBAR-SACRAL - GENERALIZED)	2	F-G-H
ARTICULAR RHEUMATISM	2	F-G
BACK PAIN - LUMBAGO	1	F-G
BEDSORES	5	F-G
CERVICALGIA - STIFF NECK	4	F-G
CONTUSIONS - SPRAINS	3	F-G
DELAY IN BONE CONSOLIDATION	3	F-G-H
EDEMA (POST-PHLEBITIC OR GENERALIZED)	5	F-G-H
FRACTURES	3	F-G-H
HEADACHES	4	F-G-H
HEALING OF TISSUES	5	F-G-H
HEAVY LEGS	5	F-G-H
INSOMNIA - NERVOUS TENSION	6	F-G
LUMBAR SCIATICA	1	F-G-H
NEURALGIA	1	F-G
OSTEOPOROSIS	2	F-G-H
PHLEBITIS	5	F-G-H
PSORIASIS	6	F-G
TENDINITIS - EPICONDYLITIS (TENNIS ELBOW)	1	F-G
VARICOSE VEINS	5	F-G-H

1 INFLAMMATORY PAINS - NEURALGIA

Here we group all those pathologies that have an inflammatory origin, acute or chronic, accompanied by painful conditions, often very intense, that make proper movement troublesome. They affect various areas of the body.

The neuralgic forms are irritant painful conditions of a sensitive nerve, usually having a sudden and violent intensity.

Those that are among the most common are tied to the sciatic nerve, to the trigeminal, and to the dental area in general.

2 PAINFUL ARTICULAR FORMS

Inflammatory rheumatic conditions, acute or chronic (*Arthritis*), which affect one or more articulations, causing strong pain and bothersome limitations to proper and normal movement, to the extent of causing deformation, at times even severe ones. They may affect various body parts and at times even affect multiple areas at once.

There are also non-inflammatory degenerative processes (*Arthrosis*) which cause tissue degeneration in one or more articulations. At first, they strike the articular cartilage, while over time they may also affect the bones (*Osteoarthritis*).

They strike one or more body parts at a time, compromising movement, with pain that may become very intense.

3 POST-TRAUMATIC PAINFUL CONDITIONS

Conditions with a varying severity, which may somewhat affect all the tissues (quite often the elements of the articulations or the bones), and which may even cause their rupture.

The therapy can be carried out even if the patient has a cast.

The typical symptoms are swelling, pain, and a sensation of heat. They affect various body parts.

4 HEADACHES AND CERVICALGIA

This group includes various disorders, very painful and bothersome, which affect the head (in various parts) and the neck, and which can even extend to the shoulders and arms. Their origin may have several causes.

5 PERIPHERAL CIRCULATORY DISORDERS - EDEMAS

There are several pathologies/dysfunctions that stem from a circulatory system that is functioning poorly, especially on a level of the lower limbs (the subjects most affected by this type of problem are without a doubt women). They may stem from inflammatory or mechanical causes and are often associated to intense pain, swelling, and limitations to movement. They may be associated to skin wounds (on the surface or deeper).

6 PROBLEMS TIED TO STRESS, SLEEP DISORDERS, AND NERVOUS TENSION

Stress and a sense of fatigue are surely amongst the most negative consequences of a wealthier society; the conditions described here and so widespread are caused by the frantic pace of work and the hectic quality that characterizes the larger part of our daily life. The consequence are a whole series of problems (which often compromise the normal day-by-day) generalized on a nervous system level (tiredness, irritability, anxiety, insomnia, "malaise", etc.) or more localized and narrowed down.

ELECTROMEDICAL WARRANTY • CONDITIONS

1. THE DEVICE AND THE RELATIVE ACCESSORIES ARE COVERED BY WARRANTY FOR MANUFACTURING FLAWS, for a period of 24 months from the date reported in the document that provides proof that the machine has been purchased. The absence of this document will entail the loss of the warranty, and any repairs (or any other type of intervention) will be the full responsibility of the customer.
2. THE TERM WARRANTY IS UNDERSTOOD AS THE REPLACEMENT OR REPAIR OF ELECTRONIC PARTS OF THE DEVICE THAT PROVE DEFECTIVE DUE TO MANUFACTURING FLAWS.
Unauthorized personnel is strictly forbidden to open the machine and, if so, entails the immediate termination of the Warranty. We moreover remind you that the warranty is revoked also in the case that the labels (with bar codes) of the Medical Device and/or its accessories are removed or destroyed.
3. THE WARRANTY DOES NOT COVER the “aesthetic” parts (*cover, bottom, battery cover, etc*) and all the parts considered of consumption/or prone to wear out, as well as the damages/breakdowns that were to be caused by negligence or carelessness during use of the device, improper installation or maintenance, interventions made by unauthorized personnel, displacement/transport performed without the due cautions, that is, pursuant to circumstances that, at any rate, cannot be directly ascribed to original flaws in the manufacture of the device by the manufacturer.
4. The Manufacturer rejects any responsibility for possible damages that may directly or indirectly stem to persons or things due to an improper use of the device, which must be used in observation of all the precautions indicated in the specific manual of the instructions for use, or in accordance with the indication/prescription of medical or health staff.
5. In the case of a fault, contact the company that has sold the machine. The device will be repaired at an authorized laboratory of the manufacturer or directly by the manufacturing company.
The device must be sent along with all the accessories and in adequate packing, together with the Warranty Certificate provided, duly stamped and filled out in all its parts. The expenses and risks of transport will be responsibility of the purchaser.
6. In case an intervention at your premises is requested, which will only be performed if technically feasible, the right to call in force will have to be corresponded, as well as the reimbursement of any possible transfer expenses (in addition to any possible repair costs, if the device were not covered by warranty).

WARRANTY CERTIFICATE

The Warranty Certificate of the medical device NOCTURNO® (all of its parts must be filled out) adheres to the Conditions summarized on this page and must be annexed to the device when sent to Technical Assistance.

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All rights reserved