GB

Backache therapy system THERAPYQUICK

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maniquick®
S W I T Z E R L A N D

GB Manual

1 Safety Instructions	1
2 Useful Information	4
3 Getting started	6
4 Operating	8
5 Miscellaneous	12
6 Conditions for service under guarantee	14

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Please fold out this page and leave it folded out
for quick reference.

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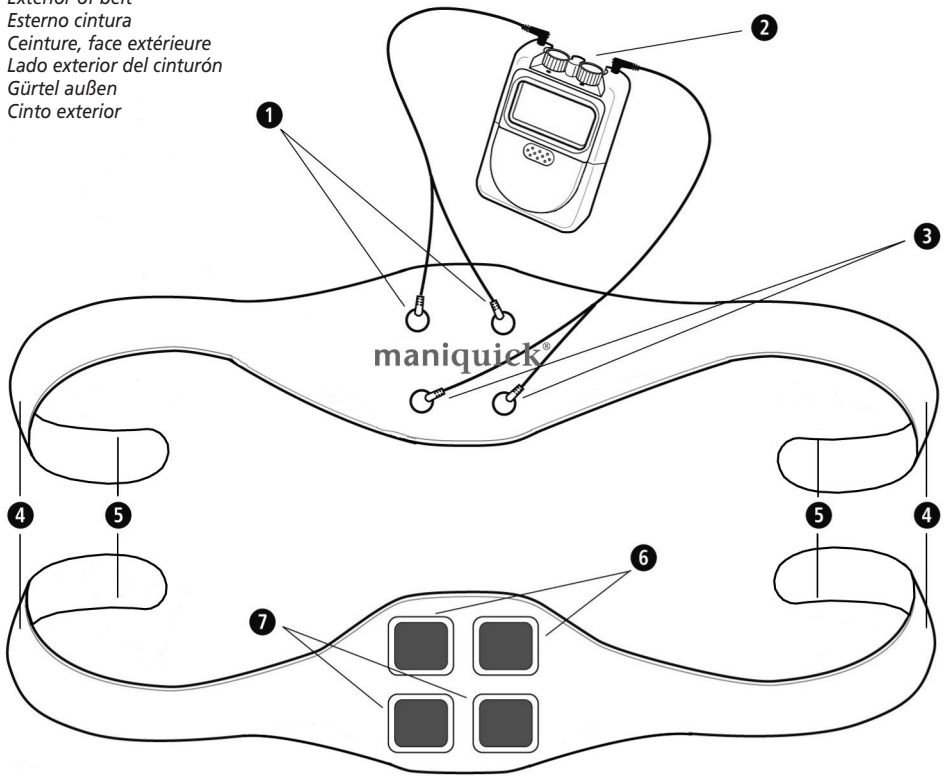
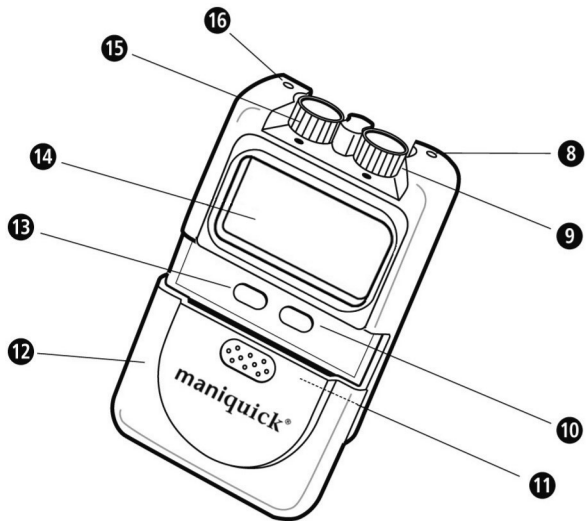
TENS unit
Apparecchio TENS
Appareil TENS
Aparato TENS
TENS-Gerät
Aparelho TENS

GB

- 1 Pushbutton connectors channel 1
- 2 TENS unit
- 3 Pushbutton connectors channel 2
- 4 belt
- 5 Velcro fastener
- 6 Electrode pads channel 1
- 7 Electrode pads channel 2
- 8 Connection socket channel 2
- 9 On/Off / Intensity channel 2
- 10 **TIMER** = therapy time setting
- 11 Battery compartment
- 12 Keypad cover
- 13 **MODE** = program selection
- 14 Display
- 15 On/Off / Intensity channel 1
- 16 Connection socket channel 1

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**Thank you
very much**

Thank you for confidence in us and congratulations!
The back pain treatment unit that you have purchased is a top quality product from **MANIQUICK**

This unit is intended for treatment of pain in the lower back area on atemporary basis according to the individual.

In order to achieve the desired effect with your MANIQUICK Backache therapy system in the long term, we recommend that you read the following information on its use and maintenance carefully.

**1.1
Important**

Please read the following safety information carefully before using the device and keep these instructions in a safe place for later use.

- Use the device only according to its intended purpose as specified in the instruction manual.
- The warranty will be invalidated if the device is used for purposes other than those for which it is intended.
- The device may only be used in conjunction with the accessories (electrodes, cable) recommended by the manufacturer.
- Do not use the device if it is not working correctly, if it has been dropped or has fallen into water or has been damaged.
- Do not attempt to repair the device yourself in the event of a malfunction since this will invalidate the warranty. Repairs should only be carried out by authorised service centres.
- Do not operate the device in the vicinity of high-frequency transmitters, e.g. microwave and short wave transmitters or radio transmitters. They may impair the device's ability to function.
- Do not switch the unit on until the belt with the electrodes has been positioned on the body.
- Do not use the device whilst taking a bath or a shower.
- Do not use the machine whilst sleeping.
- The device may not be used whilst driving a car or whilst operating and controlling machinery.
- Protect the device from moisture.
- If liquid should get into the device, remove the battery immediately and discontinue use. Should this happen, contact your specialist dealer or contact us directly.
- Never use the device in rooms where aerosols (sprays) are used or pure oxygen is being administered.

- Never remove the treatment belt ④ from the skin if the unit is still switched on. The device can be switched off at any time if necessary. Turn both intensity controllers, ⑮ and ⑨, anti-clockwise to the OFF position.

- When using the **MANIQUICK** back pain treatment unit device for professional use, a safety control has to be carried out every 2 years at an authorized service office.

WARRANTY PERIOD FOR PROFESSIONAL USER: 1 YEAR.

- **Informations about the electrodes:**

- For hygiene reasons, ensure that each user uses his or her own electrodes.
- Do not place the electrodes on the body if it is wet.
- Only use the unit on clean, dry skin.
- Do not place the electrodes to open skin parts.
- Take care never to bring the electrode pads into contact with one another. This could lead to short-circuiting of the device.
- Do not use damaged or defect lines to the electrode; there is danger of electric shock.
- Do not expose the electrode pads to direct sunlight.
- Slight reddening of the skin after use is normal and will disappear within 30 minutes. Do not use the unit again until the reddening has disappeared.
- Select shorter treatment times if skin irritations occur over a longer period of treatment.
- In case of more severe skin irritation, stop the treatment and consult a physician.
- Remove the remains of adhesives from the skin with soap and water. Do not use alcohol or solvents.

1.2

Important advice - for your health's sake

- Therapy with the **MANIQUICK** back pain treatment unit does not replace medical diagnosis or treatment. Ask your doctor for each kind of pain or disease, before using the device.
- The unit should be used **exclusively in the lower back region!** Using the unit on other parts of the body may lead to serious breathing problems, cardiovascular disturbances or cardiac arrhythmia.
- The device should not be used by the following patients:
 - patients with pacemakers, defibrillators or extreme cardiac irregularities,
 - patients with metal implants or electronic auxiliary devices,
 - diabetics and patients with abnormally high blood pressure,
 - patients with a tendency towards internal bleeding,
 - pregnant women.

- Consult your doctor if you suffer from epilepsy or certain disabilities.
- Persons, whose sensitivity is disturbed or impaired may only apply the device according to the instructions of a physician.
- Do not use the unit if suffering from period pains.
- Do not use the unit if the skin is burning or numb.
- Pain relief treatment following injury or operations may only be carried out under the supervision and instruction of a doctor or physiotherapist.
- Do not treat any parts of the body that have swellings, burns, inflammation, peeling skin, wounds or other sensitive points.
- Always seek the advice of a doctor in case of doubt and follow his instructions for treatment.
- Children may not use the device. Keep the unit out of the reach of children. Medical products are not toys!
- If the device is used from, on or near children or people who are handicapped and can only take limited use of the device, this should always be monitored.
- The treatment should be pleasant. If you feel pain or feel the application unpleasant, stop it and consult your doctor.

Please check first whether the device is complete.

1.3 Scope of delivery and packaging

The **MANIQUICK** back pain treatment unit includes:

- 1 TENS unit with belt clip
- 1 belt
- 4 electrode pads
- 2 connecting cables
- 1 9V block battery, type 6LR61
- 1 instruction manual

The packaging can be reused or recycled. Please dispose properly of any packaging material no longer required. If you notice any transport damage during unpacking, please contact your dealer without delay.

WARNING

To avoid risk of suffocation, keep packaging film away from children.

2.1 Valuable information about stimulation current

Your **MANIQUICK** back pain treatment unit is a TENS/EMS device.
TENS = Transcutaneous Electric Nerve Stimulation
EMS = Electric Muscle Stimulation
It provides treatment in the low frequency range using the TENS/EMS method. Using this method, weak electrical pulses are conducted through the skin using the electrodes thus stimulating muscles, nerve fibres and acupuncture meridians in order to utilise the body's own pain relief mechanisms.
Stimulation current is used primarily to combat pain.

2.2 Fighting pain (TENS)

TENS is understood as electrical nerve stimulation through the skin.
Pain is a natural and important warning signal of our body telling us that "something is wrong". If the reason for pain is diagnosed, but cannot or can only partially removed, a long-lasting, persistent pain evolves (= chronic pain). Now, this is in its former function as warning signal useless and can largely affect the life of the person suffering, maybe even cause aftereffects like tensions, depression or less joy of living.

Chronic pain can be treated with different methods: e.g. with medications or operations, but also with alternative, methods like acupuncture or TENS that are free of side-effects.

On the one hand, the TENS treatment can block pain signals by interrupting the transmission of impulses through the skin to the painful parts by means of electric impulses. The signal "pain" is not given on to the brain and thus not felt as pain.

On the other hand, special TENS applications activate the body's control mechanisms. Gentle electric currents make the body to release so-called "endorphins" that function as pain reducer. The pain is thus stopped, eased or removed by the body itself.

This electrical stimulation cannot remove the causes or cure the disease, but change the painful feeling. Painless conditions or conditions freed of pain can help the chronically suffering person to make new, positive experiences, from which he or she can gain power and energy.

2.3 Frequent questions and their answers

Can something happen to me, due to the current?

No, because the current strength of the electric pulses is so low that it remains completely safe even with longer applications. However, the intensity should be adjusted to the individual sensitivity.
Therefore, start for each new application with the lowest intensity level und increase it slowly according to the individual feeling. The application should never be unpleasant.

I have no experience with stimulation current therapy. How can I apply the device without problems?

The **MANIQUICK** back pain treatment unit has been particularly developed to be easily used by everyone. Even without special knowledge of the subject "stimulation current therapy" you can treat yourself safely and easily with the help of the preset programs.

How often can I use the electrodes?

According to the condition of your skin and when observing the application instructions you can use them until they lose their adhesiveness.

Can I use other electrodes as well?

No, you should only used original **MANIQUICK** electrodes.

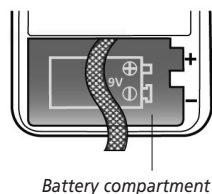
GB 3 Getting Started

maniquick®
SWITZERLANDmaniquick®
SWITZERLAND

3 Getting Started GB



3.1
Inserting / removing
batteries

1. The block battery supplied must first be inserted in the battery compartment before the unit can be used for the first time.
2. Check that the unit is switched off. Turn both intensity controllers, 15 and 9, anti-clockwise to the OFF position.
3. Open the battery compartment 11 by pushing the keypad cover 12 downwards. At the same time press down with the thumb on the area of the cover identified with raised dots.
4. Insert the battery in the base of the battery compartment taking care to match up the symbols (+ / -). Thread the textile strip under the battery; this will make it easier to remove the battery the next time it needs replacing.
5. Close the battery compartment by placing the cover on the lateral guides and sliding it upwards with slight pressure of the thumb on the area marked.
6. Follow the same procedure when replacing the battery at a later date. Always use only 9 V block batteries of type 6LR61.

! BATTERY SAFETY INFORMATION

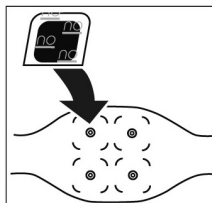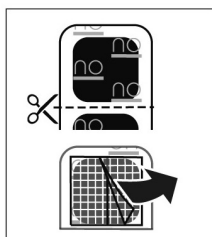- Keep away from children.
- Do not recharge.
- Do not short-circuit.
- Do not throw into fire.
- Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop.

3.2
Preparing the belt
for use

Insert pads centrally on the areas marked.

6

1. Lay the belt 4 flat on a table with the electrode side upwards so that the four pad markings are visible.
2. Lay out the electrode pads 6 and 7 and the pushbutton connectors 1 and 3 with their connection cables.
3. The unit is supplied with 4 (2 x 2) self-adhesive pads on a backing or protective film which correspond in size to the markings on the belt. Use scissors to cut carefully between the two pads on the film marked "on" without separating the pads from the film. There will now be four pads in front of you.
4. Remove the two-part protective films from the sides of pads 6 and 7 with a criss-cross pattern by pulling the films off carefully from the inside outwards. Leave the films marked "on" on the pads.
5. Position the pads 6 and 7 with the criss-cross patterned side down, centrally on the markings on the belt. Press the pads down firmly onto the belt.

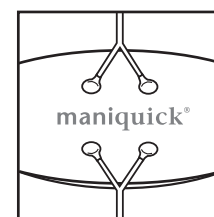
! WARNING

Only use the belt when all four pads are attached to the electrodes to prevent unpleasant electric shocks.


! NOTES

Use only the original MANIQUICK electrodes enclosed with the unit. Use separate electrode pads for each user. These may be ordered subsequently from MANIQUICK as a pad set.

For the first few applications, the electrode pads are very sticky. It is, therefore, advisable to remove the electrode pads carefully from the skin with the belt.

channel 1 channel 2

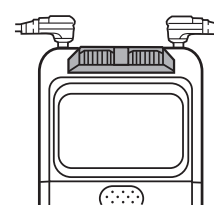6. Turn the belt over and place it flat on the table again with the self-adhesive pads facing downwards. The four pushbutton contacts are visible.
7. Take the cables and connect the two pushbutton connectors 1 to the pushbutton contacts as shown in the diagram. Do the same with pushbutton connectors 3.

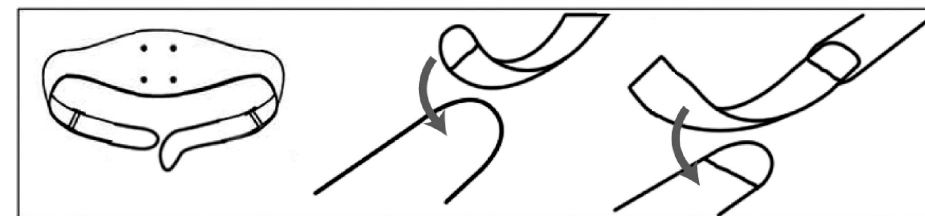
! CAUTION

Make sure that the two intensity controllers 5 and 9 are in the OFF position before inserting the cables.

8. Insert the free ends of the cables into the connection sockets of channel 1 and 2 (6 and 8) on the top of the TENS unit. It is immaterial which cable is assigned to which channel.

3.3
Adjusting the belt
width


The belt 4 is a universal size which fits everyone. The belt width may be adjusted to suit the individual by fixing the Velcro fastener 5.



GB 4 Operating

maniquick®
SWITZERLANDmaniquick®
SWITZERLAND

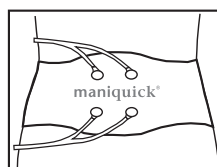4 Operating GB

4.1
Putting on the belt

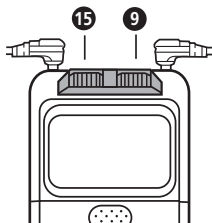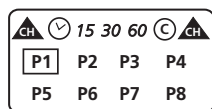1. Prior to putting on the belt, remove any residues of skin cream or ointment. The skin must be grease-free. Do not place the belt on injured or inflamed areas of skin (wounds, spots, rashes, reddening, etc.).
2. Remove the plastic films marked "on" of the electrode pads, ⑥ and ⑦. Handle these films carefully so that they can be stuck back in place to protect the pads after each use.
3. Put the belt ④ on. Position it on the area of the body to be treated. If your backache is at waist level, you will be suffering from pain in the upper treatment area. Position the belt so that the middle of the belt is at waist level.
4. If your back pain is in the lower treatment area, position the belt so that the middle of the belt sits approx. 3 - 5 cm below the waist.
5. When the belt has been manoeuvred into the correct position, bring the two ends of the neoprene straps together and push in the clip fastener ⑤ to fasten the belt tight.
6. The belt may be tightened or loosened by proceeding as described in "3.3 Adjust belt width".

4.2
Switching on the unit

1. The unit is switched on by setting the intensity controllers, ⑮ and ⑨, to the desired strength. The unit offers 8 intensity settings for treatment strength from 0 - 80 mA.
2. The unit's basic setting now appears on the display ⑭.
3. Turn the two intensity controllers on the top of the TENS unit in a clockwise direction (1 = weaker/8 = stronger) until the electric pulses of both channels can be felt. The stimulation should be perceived as pleasant.

! CAUTION

Subjective perception of the electric current may alter with changing frequencies, pulse widths or current shapes. Regulate the intensity as soon as the application is perceived as unpleasant or the pleasant prickling sensation is not felt for a longish period.

4.3
Selecting the program

The unit offers 8 preset treatment programs. The programs differ in respect of varying pulse widths and frequencies.

Program	Pulse intensity	Pulse width	Frequency	Mode
	max.			
P1	80 mA	260 µs	15 Hz	Constant
P2	80 mA	260 µs	100 Hz	Varying
P3	80 mA	260 µs	120 Hz	Constant
P4	80 mA	260-150 µs	2-100 Hz	Varying
P5	80 mA	260 µs	100 Hz	Varying
P6	80 mA	260 µs	80<->7 Hz	Varying
P7	80 mA	260 µs	120 Hz	Varying
P8	80 mA	P1 - P7		Sequential

1. The unit has been switched on by adjusting the intensity controllers, ⑮ and ⑨, to one of the 8 intensity settings (see 4.2).
2. The display ⑭ shows the unit's basic setting.
3. Push the keypad cover ⑫ downwards. Do this by pressing with the thumb on the area on the cover identified with raised dots. Press the MODE key ⑬ repeatedly until the desired treatment program (P1 - P8) is framed on the display.

NOTES

Each of these programs starts as soon as it is selected using the MODE key ⑬ regardless of whether or not a treatment time has been set. The preset treatment time on first use is 15 minutes.

Turn the two intensity controllers, ⑮ and ⑨, anti-clockwise to the OFF position to stop the program.

4.4
Selecting the treatment time

4. The most recently set treatment program is stored. The most recently used program is set and flashes when the unit is switched on again.

1. The unit has been switched on and a treatment program has been selected (see 4.2 and 4.3).
2. The display ⑭ shows the unit's basic setting with the program that has been selected. When the unit is first switched on, the display flashes for the preset treatment duration "15" or the most recently set treatment time.
3. The unit offers 4 preset treatment times, 15, 30 and 60 minutes, continuous and a setting for continuous operation (©).
4. Push the keypad cover ⑫ downwards. Press the TIMER key ⑩ repeatedly until the desired treatment time (15 - 30 - 60 min. or © for continuous operation) flashes on the display. Push the keypad cover ⑫ back up.

NOTES

The treatment time always begins immediately as soon as one of the three preset times has been selected or altered. Turn the two intensity controllers, ⑮ and ⑨, anti-clockwise to the OFF position to stop the program.

5. The appropriate display "15", "30" or "60" flashes for the duration of the treatment time set. The unit switches off automatically when the treatment time has expired. The unit does not switch off independently in continuous mode "©".
6. The most recently set treatment time is stored. The most recently selected treatment duration is set and flashes when the unit is switched back on.

4.5 Special features

- The belt may be worn comfortably under clothing when the settings for the treatment program and time have been selected and the keypad cover ⑫ has been closed again. Take care in this case that the pushbutton connectors, ① and ③, do not get pulled off and that the cables are not damaged.
- For ease of use, the TENS unit may be worn on the belt by attaching it using the clip on its rear.
- If an application is to be repeated, first turn the two intensity controllers, ⑮ and ⑨, to the **OFF** position and then back to the desired intensity level. The program is repeated with the settings selected previously.
- Try out different programs to find out which of the preset program sequences is most suitable.
- Different people experience different levels of effectiveness with a treatment program. It may be that the feeling of efficacy of a program will decrease if it has already been used for several hours at different times.
This may happen because the body has become used to the pulse intensity or the program sequence set. This may be avoided by changing the intensity and the program from time to time.

WARNING

Only use the belt when all four pads are attached to the electrodes to prevent unpleasant electric shocks. Check the condition of the electrode pads and cables regularly.

4.6 Switching off the unit

- The unit switches off automatically on expiry of the preset treatment time of 15, 30 or 60 minutes. The unit can also be switched off manually at any time. In both cases the intensity controllers, ⑮ and ⑨, must be turned anti-clockwise until they click into the OFF position and the display goes blank.

WARNING

To prevent unpleasant electric shocks, never remove the treatment belt ④ from the skin whilst the unit is still switched on. First turn the two intensity controllers, ⑮ and ⑨, anti-clockwise until they click into the OFF position and the display goes blank.

- Leave the pads on the electrodes on the belt. For safe storage, place the plastic films marked with "on" back on the electrode pads after each use. This will increase the service life of both electrodes and pads.

4.7 Troubleshooting

Always check the unit and accessories each time before use to prevent damage and defects:

1. The battery has sufficient charge and is not corroded.
2. All pads stick fast and centrally to the electrodes. They are not damaged or dried out.
3. The cables are not damaged and are in good condition.
4. The contact connectors of the cables are properly and firmly attached to the pushbutton contacts on the belt.
5. The cables fit tightly into the connection sockets on the TENS unit.

The following table may help if a fault occurs despite these precautions. Please contact your service centre if it is not possible to remedy a fault in the manner described.

Fault	Cause	Remedy
The unit cannot be switched on.	No battery in the device or battery spent.	Replace battery.
	Battery not inserted correctly.	Insert battery again.
The unit can be switched on but switches off again immediately.	Battery spent.	Replace battery.
The unit is switched on but is not generating any electric pulses.	Cable broken.	Replace cable.
	Cable not corrected properly.	Connect cable properly.
	Treatment time has expired.	Switch unit to OFF position and switch back on.
The unit cannot be switched on although a new 9 Volt battery has been inserted.		Contact the service centre.

**5.1
Cleaning and
Maintenance**

*A safety check must
be carried out at an
authorised service centre
every 2 years if the
MANIQUICK back pain
treatment unit is used
commercially.*

TENS unit:

- The device does not require any special maintenance.
- Remove the battery before cleaning the device.
- Never use aggressive cleaning products or stiff brushes.
- Clean the device with a soft, possibly slightly moistened cloth. Do not allow water to penetrate the device. Do not use the device again until it is completely dry.
- Take the battery out of the unit if it is not going to be used for more than a week.
- Do not expose the device to direct sunlight and protect it against dirt and moisture.
- Store the unit together with the belt, cables and pads covered by the protective films in a clean, dry place. The pads should remain on the electrodes on the belt and be rolled up together.

Belt and pads:

- If the belt has become damp during use, allow it to dry naturally before using it again.
- Take the pads off the belt before cleaning it. Ensure that the pads are covered on the skin side by the protective films marked with "on". Also detach the cables and the TENS unit from the belt. Do not pull on the cables but on the connectors attached to the ends of the cables.
- The service life of the electrode pads depends on their storage, skin type and use. The electrode pads must be replaced with new ones if the stickiness of the self-adhesive pads decreases significantly, if they dry out or are if they are very dirty.
- Wash the belt in cold water by hand. Do not wash the belt in the washing machine. Lay the belt out flat or hang up to dry naturally. Do not use a tumble drier.

Cables:

- Remove the cables from the belt and the TENS unit for cleaning. Do not pull on the cables but on the connectors attached to the ends of the cables.
- Only clean the cables with a clean, lightly moistened cloth and do not use them again until they are completely dry.

**5.2
Disposal**

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner.

Please remove the battery before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

**5.3
Accessories
MANIQUICK
Back Pain
Treatment Unit**

Ask in your specialist shop or directly at our hotline.:

set of cables item number: MQ.861.000
set of pads (comprising 4 pads) item number: MQ.862.ADS

**5.4
Technical Data**

Item # MQ.860.000:	THERAPYQUICK Back Pain Treatment Unit
Dimensions (L x W x H) :	approx. 96 x 65 x 35 mm (TENS unit)
Weight :	approx. 150 g (incl. battery)
Channels :	2
Initial pulse intensity :	max. 80 mA (at 0.5 kΩ) 68 mA (at 1 kΩ) 46 mA (at 2 kΩ)
Voltage :	max. 110 V (open) 40 V (at 0,5 kΩ) 68 V (at 1 kΩ) 92 V (at 2 kΩ)
Current density at : the electrodes	max. 1,95 A/cm ² x 10 ⁻⁵ , The MANIQUICK electrodes are suitable for these values
Frequency :	15 - 120 Hz
Pulse width :	150 - 260 µs
Curve shape :	two-phase asymmetrical
Programs :	8
Power supply :	1 x 9V block battery Type 6LR61
Operating conditions :	0 °C to +40 °C, 20 to 90 % relative humidity
Storage conditions :	-10 °C to +60 °C, 20 to 93 % relative humidity

This unit is certified in accordance with EC Directives and provided with the CE mark (conformity mark) "CE0434". It complies with European Standard EN 60601-1/1990.

Right reserved to make technical and design changes during
the course of continuous product improvement.

GB

6 Conditions for service under guarantee

maniquick®
S W I T Z E R L A N D

1. **MANIQUICK SA** guarantees the absence of faults and mechanical defects and the completeness of the product when it is sold.
2. Repairs can be done only at Service Centres authorized by **MANIQUICK SA** or by the Authorized Distributor in the country where you have purchased the appliance.
3. The duration of the period of guarantee on the materials and on manufacturing and operating defects is 5 years* with effect from the date of sale, subject to all the conditions for the Purchaser shown in the Instructions for use. The obligations of guarantee do not cover expendable materials: electrode cups, elastic bands, power supply cables etc..
4. Complaints relative to the product must be presented directly to the authorized service centre. The service centre, on the basis of an agreement with the Vendor, is competent to establish the cause and nature of the fault. The product will be replaced only if it cannot be repaired.
5. The owner of the product will provide autonomously for delivering and collecting the product from the place of the repair under guarantee.
6. The time necessary for the repair under guarantee will be subject to the degree of seriousness of the fault.
7. The responsibility of MANIQUICK SA under these conditions of guarantee will be limited to the amount paid by the Purchaser for the product in question.

* 1 YEAR WARRANTY FOR PROFESSIONAL USERS

The guarantee on the product will not be applied in the following cases

- certificate of guarantee filled in incorrectly or falsified; serial number (if applicable for the specific product) damaged or removed;
- **absence of commercial and financial documents (cash receipt, delivery note) proving the purchase;**
- opening or repair of the product by the Purchaser or by an unauthorized Service Centre;
- defects following failure by the Purchaser to observe the rules for use, storage and transport of the product shown in the relative technical and regulations documentation, in which case the regulations of the aforementioned documentation which have been infringed must be specified;
- penetration of foreign bodies or liquids inside the product;
- presence on the product of internal or external mechanical defects (cracks, signs of a fall, chipped parts, etc.) consequent to the incorrect use, installation or transport of the product;
- components or accessories subject to rapid wear and tear (electrode cups, elastic bands);

maniquick®
S W I T Z E R L A N D

6 Conditions for service under guarantee

GB

- defects caused by external agents (dirt visible externally and internally, traces of corrosion);
- defects due to causes independent of the manufacturer, such as power supply voltage step changes, natural phenomena or disasters, fires, actions by animals, insects etc.
- complaints for defects found by the Purchaser that are made after the expiry of the guarantee period or any other period established by the law for reporting any defects;
- use of the product by the Purchaser not for personal (domestic) use but for commercial and profit-making purposes.

Conditions for the return of a defective product

Defective products will be accepted only if clean and complete, in the original packaging, together with the certificate of guarantee correctly filled in and the commercial and financial documents (cash receipt of the store, delivery note) that prove the purchase. Please attach a description of the problem found to the appliance

I have read the conditions of guarantee. I receive the product in working order and I have no complaint to make regarding its external appearance and completeness.

Purchaser's Signature:

Address of the point of acceptance for service under guarantee:

Please consult:
www.maniquick.com
 e-mail: info@maniquick.com
 Tel. +39 02 90.39.00.38
 Fax +39 02 90.39.02.79

Before sending the appliance, please contact us by phone or e-mail to agree on how the product is to be returned.

GB

maniquick®
S W I T Z E R L A N Dmaniquick®
S W I T Z E R L A N D

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I

Send this coupon in case of repairs:

Product: TherapyQuick

Serial Number:

(see bottom of the unit)

Date of purchase: / /

Dealer's Stamp:

Buyer's Full Name:

Street/Square: N°:

City and State: Postal Code:

Country:

Phone Number:

E-mail: @

Problem description:

.....

.....

.....

Signature:

Date: / /

WARRANTY IS VALID ONLY IF ACCOMPANIED BY INVOICE/TICKET.

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